INSTRUZIONI PER L'USO NOTICE D'UTILISATION GEBRAUCHSANWEISUNG

PRIMA® HOME TEST

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FOOD INTOLERANCES (FI)

Food Intolerance (FA) is a generic term used to indicate a variety of reactions to one or more food types.

The intolerance manifests itself when the body is no longer able to digest and therefore assimilate a given substance. There can be a variety of symptoms, the most common of which are digestive problems such as stomach ache, flatulence, diarrhoea and constipation.

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PRECAUTIONS

- 1) Carefully read the instructions before taking the sample.
- 2) Keep out of reach of children.
- 3) The kit shall be used by adults or at least under the supervision of an adult.
- 4) The lancet is monouse and must not be re-used by two people.
- 5) Do not use the kit after expiry date.
- 6) Store between +4 °C and +30 °C. Do not freeze.
- 7) External use only. DO NOT SWALLOW.
- 8) After use, dispose of all components in accordance with local regulations, seek advice from your pharmacist.

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CONTENTS

THE BOX CONTAINS ALL THE NECESSARY TO COLLECT AND MAIL THE SAMPLE:

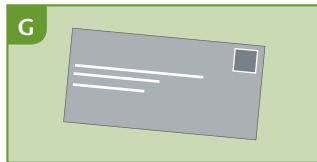
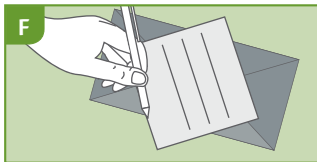
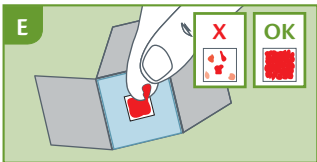
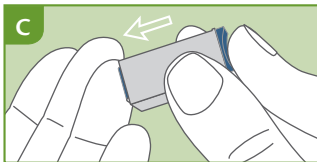
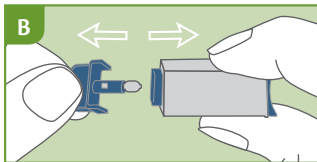
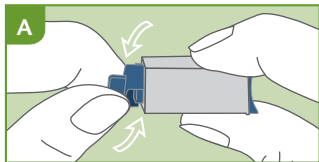
- 1 absorbent card for blood sampling
- 1 disinfectant pad
- 2 sterile lancets for blood sampling
- 1 plastic bag with zip
- 1 Instruction for use
- 1 customer information form
- 1 pre-printed envelope

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PROCEDURE



- 1) Clean the fingertip by washing carefully the hands.
- 2) Tear the protective pouch (from the notch) and take out the absorbent pad.
- 3) Hold the lancet and carefully rotate 360° the protective cap without pulling it.- **FIG. A**
- 4) Extract and dispose the released cap. - **FIG. B**
- 5) Massage the end of the finger, from the base of the finger to the end, to enhance the blood flow.
- 6) Press the lancet, on the side from where the cap was extracted, against the fingertip (it is advisable the ring finger side). - **FIG. C** The tip retracts automatically and safely after use.
- 7) Help the spillage of blood by applying pressure on the fingertip. - **FIG. D**
- 8) Place 4/5 drops of blood in the square on the card without pressing your fingertip on the membrane - **FIG. E**. Disinfect the finger with the disinfectant gauze provided. **Warning:** the blood must almost entirely cover the area of the absorbent membrane.
- 9) Fill the form included with the kit in every part and sign it. - **FIG. F**
- 10) Put the the absorbent pad card into the plastic bag supplied, close the zip and ship together with the form into the pre-printed envelope included in the kit. Mail it within the same day. - **FIG. G**
- 11) After the laboratory has analyzed your sample, the report with the results will be sent to you, along with nutritional advices.

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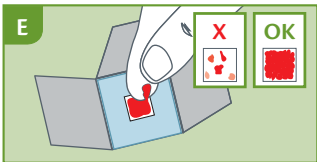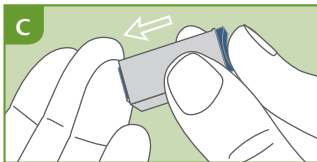90 FOODS

Albumen	Cauliflower	French bean	Oats	Sheep's milk
Almond	Celery	Garlic	Olive	Sole
Ananas	Cephalopods mix	Grapefruit	Onion	Soy
Anchovy	Chard	Grapes	Orange	Spinach
Apple	Cherry	Hake	Parsely	Strawberry
Apricot	Chicken	Hazelnut	Pea	Sugar cane
Artichoke	Chickpea	Honey	Peach	Sunflower
Asparagus	Cinnamon	Kiwi	Peanut	Sweet pepper
Baking powder	Clam	Kola nut	Pear	Tea
Banana	Cocoa	Lamb	Pepper	Tomato
Barley	Coconut	Lemon	Plum	Trout
Bean	Cod	Lentil	Pork	Tuna
Beef	Coffee	Malt	Potato	Turkey
Berries mix	Corn	Mandarin	Pumpkin	Walnut
Brewer's yeast	Cow's milk	Marjoram	Rabbit	Watermelon
Broccoli	Crustaceans mix	Melon	Rice	Wheat
Cabbage	Cucumber	Mushrooms mix	Rye	Yogurt
Carrot	Eggplant	Mustard	Salmon	Yolk

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F.A.Q. - QUESTIONS AND ANSWERS

IS IT TRUE THAT FOLLOWING A DIET IN WHICH THERE ARE NO FOOD INTOLERANCE YOU CAN LOSE WEIGHT?

It has been observed that the food intolerance can cause disturbances to metabolism with changes in weight both in excess and in defect. By removing these foods and then controlling the intake the metabolism will be activated with a consequent loss of body mass.

IS IT POSSIBLE FOR THE FOOD INTOLERANCES TO CAUSE A WIDE RANGE OF DISORDERS?

Yes. Food intolerances cause a state of inflammation that can spread to reach various organs. In addition, all inflammatory conditions may worsen due to ongoing intolerance.

I HAVE HEARD ABOUT FOOD ALLERGY AND FOOD INTOLERANCE. ARE THEY THE SAME?

No. Allergies are related to the production of class IgE immunoglobulin and symptoms appear within a

short period of time from the assumption of the food (urticaria, disorders of the mucous membranes, nausea, vomiting, etc.). Food intolerance is related to the production of IgG immunoglobulin. The disorders are manifested in a long time with a tendency to become chronic.

IF I AM INTOLERANT TO A FOOD, SHOULD I ELIMINATE IT FOREVER FROM MY DIET?

No. Intolerance to a food can be of three different levels: First, Second and Third. The test result specifies to what degree is your intolerance for a certain food. Together with the outcome of the test, you will be given specific guidance for its gradual reintroduction into the diet.