## THE AMNIOTIC LEAK SELF-TEST DETECTOR

**AMNIO  
CHECK**
**Amniotic Leak Self-Test Detector**
*Selbsttest-Detektor  
für auslaufendes Fruchtwasser*
**Détecteur d'auto-test  
de fuite amniotique**
*Self-test per la rilevazione di perdite  
di liquido amniotico*
**ISTRUZIONI PER L'USO  
NOTICE D'UTILISATION  
GEBRAUCHSANWEISUNG**

The AMNIOCHECK detector is an in vitro self-test, easy-to-use home test kit that allows you to detect probable leakage of amniotic fluid whenever your physician suspects you may be in a higher risk than normal of amniotic fluid leak. Risk of amniotic fluid leak is higher than normal during 48 hours after amniocentesis tests, or when encountering unidentified wetness, or among women with history of premature membrane rupture. The AMNIOCHECK changes colors when it comes in contact with fluid of pH levels greater than or equal to 6.5 units. Amniotic fluid pH levels are greater than 6.5.

The AMNIOCHECK gives you a little extra peace of mind — when you need it most.

## PRINCIPLE OF OPERATION

Simply attach the special AMNIOCHECK to your underwear and get on with your day. When you feel a fluid leak check for a color change on the yellow background. The pad color change can be partial or complete (any intensity, shape, size or location). Note: a color change appears within

15 minutes following a fluid leak. If there is a color change to blue or green, amniotic fluid may be leaking from your uterus, or you may have a vaginal infection.

You should consult your physician or go to the hospital without delay.

If the panty liner remains yellow or turns yellow after 15 minutes, the fluid leak is probably urine, which is common during pregnancy. Any minimal amounts of amniotic fluid leakage which can be sensed as wetness by the user will create a visible blue or green stain. For further diagnosis and medical care, report the results to your physician.

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- Confirm that your water has broken so that you arrive at the hospital in time for a safe delivery.
- Identify a possible membrane rupture, especially after amniocentesis.
- Prevent complications or premature birth.

## INDICATIONS FOR USE

This product is intended to detect leaking amniotic fluid and identify cause of wetness during pregnancy. For further diagnosis and medical care, report or show test results to your physician.

If you suspect you have a vaginal infection, consult your physician.

## LIMITATION OF THE DEVICE

Antibiotic therapy of vaginal infections (such as bacterial vaginosis or trichomoniasis) can lead to an elevated vaginal pH level, which may result in a false-positive test result for the presence of amniotic fluid. The possibility of a vaginal infection may be evaluated using standard diagnostic procedures.

## STORAGE

Store in a dry place at room temperature. Keep the AMNIOCHECK sealed until use to avoid contamination. Please note the expiration date stated on the AMNIOCHECK. If in doubt,

keep out of reach of children. For single use only. Do not use the device after the expiry date or if the pouch is damaged.

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Se il salvastip è giallo o torna giallo dopo 15 minuti, si tratta probabilmente di una perdita di urina, which is common during pregnancy. Any minimal amounts of amniotic fluid leakage which can be sensed as wetness by the user will create a visible blue or green stain. For further diagnosis and medical care, report the results to your physician.

Amniotic fluid increases the risk of infection to both mother and fetus and early detection can help reduce the risk. AMNIOCHECK is recommended for screening either high-risk or normal pregnancies. Leaking

AMNIOCHECK recommends you to see your doctor if you have a vaginal infection, consult your physician. If you suspect you have a vaginal infection, consult your physician.

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## PACKAGE CONTENTS

## THE AMNIOCHECK CONTAINS:

- 5 test panty liners, and instruction for use (ref. 100088-5) or
- 30 test panty liners, and instruction for use (Ref. 100088-30)

Keep the AMNIOCHECK sealed and use it immediately after opening.

## CONDITIONS FOR USE

To ensure valid results, do not use the AMNIOCHECK device if the answer to any of the following questions is YES:

- 1) Is it less than 12 hours after sexual intercourse?
- 2) Is it less than 12 hours after vaginal douching?


During bleeding results reading may need professional assistance. Report all bleeding situations to your physician.

Disregard positive result and retest if there is a possibility of wetness/ wetness remains from shower, bath etc, as these may interfere with the results.

In case of sensitization or irritation of skin, at the application site, stop use and contact your physician

## DIRECTIONS FOR USE

- 1) Tap water can interfere with the test – wash and dry your hands.
- 2) Unwrap the AMNIOCHECK panty liner. The panty liner contains a pale yellow indicator.
- 3) Attach the panty liner to your underwear. The pale yellow indicator strip should sit directly against your vagina. Wear the panty liner as you would a normal panty liner until you feel wetness (do not exceed 12 hours). **Note:** use the panty liner while sitting, standing or walking. Do not use the panty line while lying down.
- 4) When you feel a fluid leak, remove the panty liner and wait 15 minutes. Then check for a color change on the panty liner's yellow background. Also when replacing with a new panty liner, check for a color change.
- 5) If you feel wetness and the panty liner is completely yellow, after 15 minutes, consider using another test.
- 6) Dispose the panty liner as you would have disposed other hygiene products.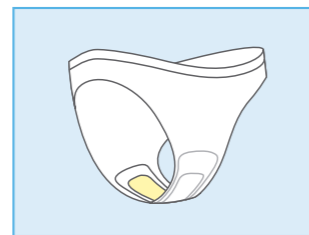

## RESULTS INTERPRETATION

## 15 MINUTES AFTER REMOVAL LOOK FOR A COLOR CHANGE ON THE PANTY LINER.


Result reading should be done in a well illuminated place.

## PROBABLE AMNIOTIC FLUID LEAK


If there is a color change, partial or complete, to blue or green (any intensity), in any size, shape or location, on the panty liner, the fluid leak is probably amniotic fluid. The color change can look cloudy and can be more intense along part of the yellow strip edge. In any such case, consult your physician or go to the hospital without delay.

**Important:** The indicator strip may also turn blue or green if vaginal infection is detected. If the indicator strip turns blue or green, always consult your physician.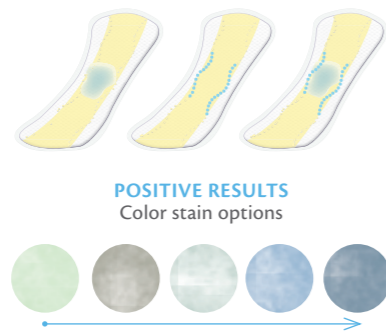

## PROBABLE NEGATIVE RESULT

If the panty liner did not change color (remained yellow or turned yellow after 15 minutes), the vaginal wetness was probably not caused by amniotic fluid. If you continue to experience vaginal wetness, apply a new panty liner and repeat the test and consult your physician.

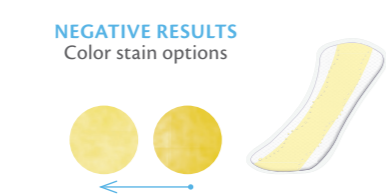**Warning:** In case of heavy leaks or the suspect of broken waters, please seek medical advice immediately, even if the device does not change color.

## PERFORMANCE DATA

A total of 232 pregnant women were enrolled in a pivotal study of the AMNIOCHECK Test Kit. The AMNIOCHECK, as a continuance test, demonstrated a sensitivity of 97.06% (99/102), with a 95% CI of [91.64%;99.39%] and specificity of 96.92% (126/130), with a 95% CI of [92.31%;99.16%].

- 1) Bornstein J, Geva A, Salt I, Fait V, Schoenfeld A, Shoham HK, Sobel J. Noninvasive diagnosis of premature ruptured amniotic membranes using a novel polymer. Am J Perinatol. 2006 Aug; 23(6):351-4.
- 2) Bornstein J, Ohel G, Sorokin Y, Reape KZ, Shnaider O, Kessary-Shoham H, Ophir E. Effectiveness of a novel home-based testing device for the detection of rupture of membranes. Am J Perinatol. 2009 Jan;26(1):45-50.

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