Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
+41 91 605 1030
support@primalabsa.ch
primalabsa.ch

EC REP Qarad EC-REP BV
Pas 257
2440 Geel - Belgium

820011_IFU_11_5.0 03/2022

REF 820011

Registered trademarks.

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

MENOPAUSE FSH TEST

Self-Test for the determination of the Follicle Stimulating Hormone - FSH in urine
Selbsttest zum Nachweis des Follikelstimulierenden Hormons - FSH im Urin
Autotest pour la détection de l'hormone folliculo-stimulante - FSH dans l'urine
Self-test per la determinazione dell'ormone follicolo-stimolante - FSH nelle urine

PRIMA
HOME TEST

ENGLISH THE FSH HORMONE AND MENOPAUSE

The menopause is caused by a modification of the hormonal balance in woman's body. This change arises generally when close to 45 years old. But true signs are observed around 55 years old. A menstrual cycle duration getting irregular is the early indication of the beginning menopause (peri-menopause). Menopause is the consequence of the end of ovary follicle life-cycle leading to increase of Follicle Stimulating Hormone (FSH) circulating level. Concentration of FSH increases during menopause passing from values less than 20 mIU/mL up to approximately 80 mIU/mL.

MENOPAUSE FSH TEST can detect the presence of FSH in urine as soon as the concentration goes up from 25 mIU/mL and over.

TEST PRINCIPLE

MENOPAUSE FSH TEST is an immunochromatographic test that detects FSH through special gold-conjugated monoclonal antibodies included in the reactive strip.

REFERENCES


1. Robert J. Norman "Fertility testing" *Australian Prescriber*, 2002;25:38-401
2. Buckler H. "The menopause transition: endocrine changes and clinical symptoms." *J Br Menopause Soc.* 2005 Jun;11(2):61-5.
3. National Collaborating Centre for Women's and Children's Health (UK) "Menopause: Full Guideline". 2015 Nov. National Institute for Health and Care Excellence: Clinical Guidelines.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * hermetically sealed aluminium pouches containing each:
1 device for the MENOPAUSE FSH TEST including a protective cap
1 desiccant bag

- * 1 instructions for use leaflet

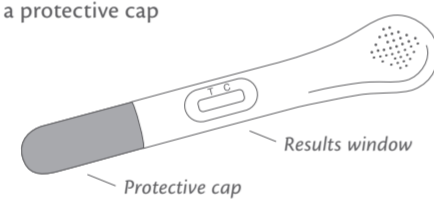- * Do not open the sealed aluminium pouch until just before performing the test. Take care to open it as marked.
- * The desiccant bag must not be used. Dispose of it with household waste without opening it.
- * Material required but not supplied: a device to measure time (i.e. timer, watch).

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PRECAUTIONS

1. Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
2. Keep the Test out of the reach of children.
3. Do not use the Test after the expiry date or if the package has been damaged.
4. Follow the indicated procedure and time strictly.
5. Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
6. Use the test once only.
7. The test is for external use only. DO NOT SWALLOW.
8. In vitro diagnostic device for individual use.
9. After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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