ENGLISH

DIRECTIONS FOR USE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag.
3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it.
4) Carefully massage the finger chosen for the puncture...
5) Holding the hand down, massage the finger until a large drop of blood forms.
6) Take the pipette without pressing the bulb.
7) Deposit the blood collected into the Specimen well (S) of the cassette.
8) Wait for the blood to be totally dispensed in the well.
9) Wait for the colored line(s) to appear. Read results at 10 minutes.

RESULTS INTERPRETATION

Please refer to the illustration and compare the T line intensity with "Vitamin D Color card" provided with the kit. As control system, a colored line will always appear in the control line region indicating that proper volume of specimen has been added and membrane wicking has occurred.

DEFICIENT 0-10 (ng/ml) 0-25 (nmol/l)
INSUFFICIENT 10-30 (ng/ml) 25-75 (nmol/l)
SUFFICIENT 30-100 (ng/ml) 75-250 (nmol/l)
EXCESS >100 (ng/ml) >250 (nmol/l)
INVALID
Includes diagrams of test results and a 10 min. timer icon.

FA.Q. - QUESTIONS AND ANSWERS

HOW DOES THE VITAMIN D TEST WORK? In how many, a 25-hydroxy vitamin D is the main storage form of vitamin D in the body. Therefore, the overall status of vitamin D can be determined by detecting the content of 25-hydroxy vitamin D.
WHEN SHOULD THE TEST BE USED? The clinical application of vitamin D is mainly for diagnosis, treatment and monitoring of rickets (children), osteomalacia, postmenopausal osteoporosis and renal osteopathy.
CAN THE RESULT BE INCORRECT? The results are accurate as far as the instructions are carefully respected.

HOW TO INTERPRET THE TEST IF THE COLOR AND THE INTENSITY OF THE LINES ARE DIFFERENT? Please refer to the illustration and compare the T line intensity with "Vitamin D Color card" provided with the kit.
WHAT DO I HAVE TO DO IF THE RESULT IS DEFICIENT OR INSUFFICIENT? If the result is deficient or insufficient, it means that the vitamin D level in blood is less than 30 ng/mL and that you should consult a physician to work the test result.
WHAT DO I HAVE TO DO IF THE RESULT IS SUFFICIENT? If the result is sufficient, it means that the vitamin D level is higher than or equal to 30 ng/mL and is within the normal range.

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