Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND

support@primalabs.ch
primalabs.ch

EC REP Qarad EC-REP BV
Pas 257
2440 Geel - Belgium

800017_IFU_11_5.2 05/2023

REF 800017

Registered trademarks.

INSTRUCTIONS FOR USE GEBRAUCHSANWEISUNG NOTICE D'UTILISATION ISTRUZIONI PER L'USO

HELICOBACTER PYLORI TEST

Self-test for the determination of anti Helicobacter pylori antibodies in whole blood samples

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ENGLISH HELICOBACTER PYLORI INFECTION AND SPECIFIC IgG

Helicobacter pylori is a bacterium whose ideal habitat is the human gastric mucosa. The infection is often asymptomatic, but it can sometimes cause gastritis and ulcers in the stomach or in the duodenum segment of the small intestine. The most common symptoms of these conditions are heartburn or pain in the upper part of the abdomen, especially if fasting or in the early morning, when the stomach is empty. Other associated symptoms include nausea, feeling full, reflux, loss of appetite, burping, weight loss and diarrhoea. Ulcers can sometimes bleed, causing anemia over long periods of time. In the long term, H. pylori infection is associated with an increase in the risk of gastric cancers. The simplest way to detect infection is by checking for the presence of specific antibodies (IgG) that are produced by the immune system after coming into contact with the bacterium and trying to eliminate it. The Helicobacter pylori Test can detect the presence of these antibodies in a few minutes.

PRINCIPLE OF THE TEST

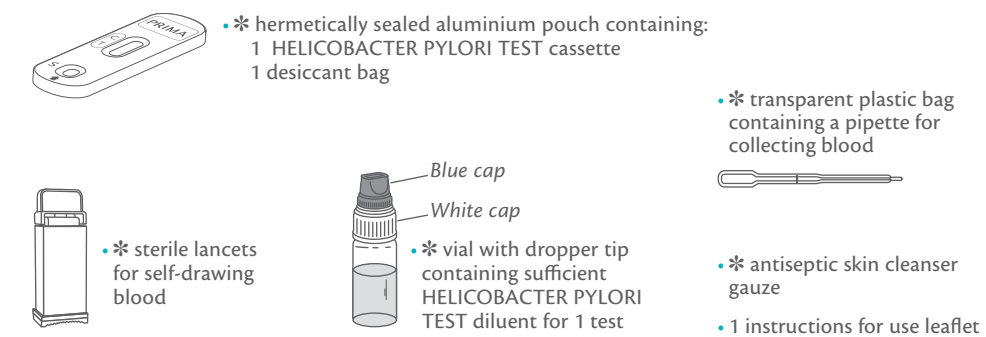
Helicobacter pylori Test is an immunochromatographic test that detects the presence of anti-Helicobacter pylori IgG antibodies in the blood, using special monoclonal antibodies conjugated with gold and integrated into the reactive strip.

REFERENCES

- Mati Moyat, Dominique Velin. *Immune responses to Helicobacter pylori infection.* World J. Gastroenterol. 2014 may 21; 20(19): 5583-5593IgE." Journal of Allergy and Clinical Immunology. 2014;133(2):589-91.
- Khean-Lee Goh, Wah-Kheong Chan, Seiji Shiota and Yoshio Yamaoka. *Epidemiology of Helicobacter pylori Infection and Public Health Implications.* Helicobacter, 2011 Sep;16(0 1):1-9.
- Kyichi Adachi, Tomoko Mishiro, Shino Tanaka and Yoshikazu Kinoshita. *Analysis of negative result in serum anti-pylori IgG antibody test in cases with gastric mucosal atrophy.* J. Clin Biochem Nutr.2016 Sep;59(2):145-148.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
- * Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
- * The desiccant packet must not be used. Dispose of it with household waste without opening it.

PRECAUTIONS

- Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
- Keep the Test out of the reach of children.
- Do not use the Test after the expiry date or if the package has been damaged.
- Follow the procedure exactly, using only the specified quantities of blood and diluent.
- Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
- Use the test and lancing device once only.
- The test is for external use only. DO NOT SWALLOW.
- In vitro diagnostic device for individual use.
- Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
- After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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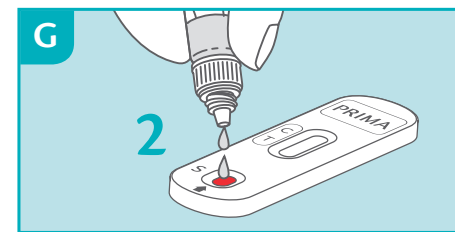
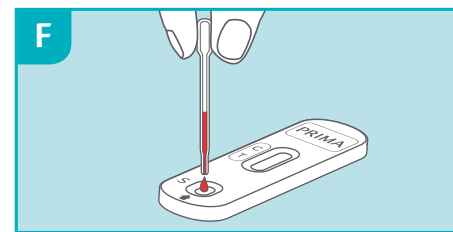
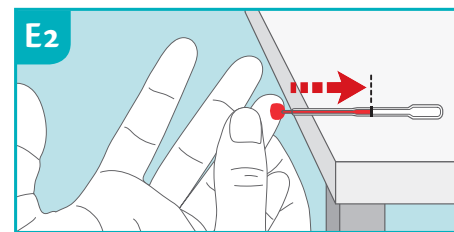
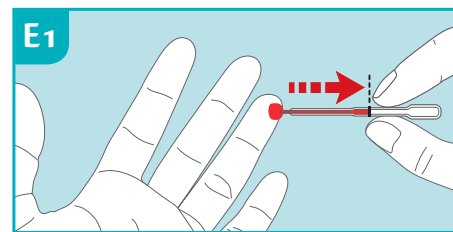
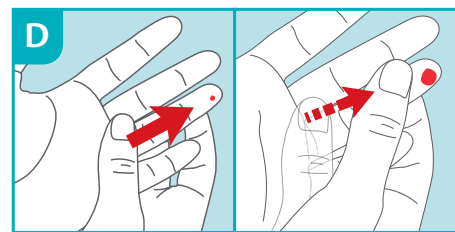
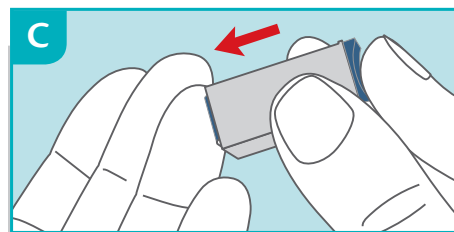
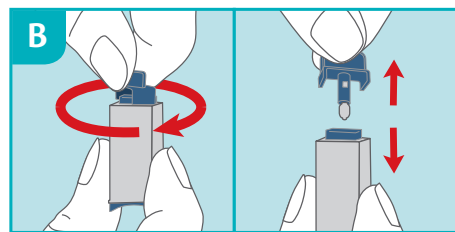
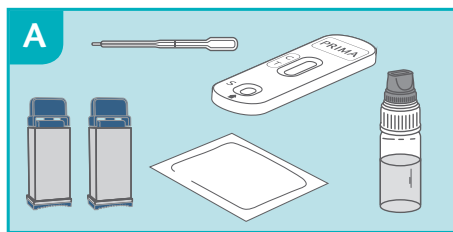
ENGLISH TEST PROCEDURE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag.
3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it.
4) Carefully massage the finger chosen for the puncture...
5) Holding the hand down, massage the finger until a large drop of blood forms.
6) Take the pipette without pressing the bulb.
7) Place the blood collected with the pipette into the well indicated on the cassette.
8) Unscrew the blue cap from the dropper vial.
9) Wait 5 minutes and read the results referring to the next section for result interpretation.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. This result means there are anti-Hp antibodies in the blood and you must consult a physician.

NEGATIVE RESULTS

A coloured band appears only under the C (control) sign. This result means that the presence of anti-Hp antibodies cannot be detected in the blood.

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new blood sample.

F.A.Q. - QUESTIONS AND ANSWERS

HOW DOES THE HELICOBACTER PYLORI TEST WORK? IgG antibodies are produced in high quantities by our body in response to infections.
WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the level of anti-Helicobacter pylori specific antibodies belonging to the IgG class is equal to or higher than 15 AU/mL.

result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the test band.

IS THE RESULT RELIABLE IF READ AFTER 5 MINUTES? No. The test must be read 5 minutes after the procedure is completed.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the level of anti-Helicobacter pylori specific antibodies belonging to the IgG class is equal to or higher than 15 AU/mL.
WHEN CAN THE TEST BE USED? HELICOBACTER PYLORI TEST is recommended when the typical symptoms of Helicobacter pylori gastric infection are present, such as stomach pain, heartburn, nausea, feeling full, burping, loss of appetite, diarrhoea and weight loss.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally spilt, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the

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