OVULATION LH TEST

Self-test for detecting the luteinizing hormone (LH) in urine

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During a woman's fertile age (from puberty to menopause), her reproductive system will go through a series of important structural and functional changes every month. This occurs as a result of the continuous variations in the levels of oestrogens and progesterone. These hormones are regulated by the hypothalamic-pituitary-gonadotropin axis and are responsible for the menstrual cycle, which has 4 phases, i.e. menstrual phase, luteal phase, follicular phase, and luteal phase. During the ovulation phase, which occurs around day 17/18 of the menstrual cycle, the level of oestrogen in the blood is so high that the pituitary gland starts to produce fertilised. The egg can be fertilized for the 24 hours following release.

Measuring LH levels in the urine is therefore an effective way to optimise the chances of conception. The OVULATION LH TEST allows you to detect a level of LH above 25 mIU/ml and therefore identify the days you are most fertile.

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

* Do not open the sealed aluminium pouch until just before performing the test. Take care to open it as marked. The desiccant bag must not be used. Dispose of it with household waste without opening it.

* Material required but not supplied: a device to measure time (i.e. timer, watch).

PRECAUTIONS

- Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
- Keep the Test out of the reach of children.
- Do not use the Test after the expiry date or if the package has been damaged.
- Follow the indicated procedure and time strictly.
- Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
- Use the test once only.
- The test is for external use only. **DO NOT SWALLOW.**
- In vitro diagnostic device for individual use.
- After using, please dispose of all components according to your local waste disposal laws.
- Ask your pharmacist for advice.

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BIBLIOGRAFIE

- Su HW, Yi YC, Wei TY, Cheng TC, Cheng CM "Detection of ovulation, a review of currently available methods"
- Bioeng Transl Med. 2017 May 16;2(3):238-246.
- Leiva RA, Bouchard TP, Abdullah SH, Ecochard R.; "Urinary Luteinizing Hormone Tests: Which concentration threshold best predicts ovulation?" *Front public health* 2017, 5: 320.

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