Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
support@primallabs.ch
primallabs.ch

EC REP Qarad EC-REP BV
Pas 257
2440 Geel - Belgium

800034-IFU-11.5.1-01/2023

REF 800034

Registered trademarks

INSTRUCTIONS FOR USE GEBRAUCHSANWEISUNG NOTICE D'UTILISATION ISTRUZIONI PER L'USO

C-REACTIVE PROTEIN TEST

Rapid self-test for the semi-quantitative detection of C-Reactive Protein (CRP) in whole blood samples

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Sterilance Medical (Suzhou) Inc.
No.168 PuTuoShan Road
New District
215133 Suzhou, Jiangsu - P. R. China

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Lot number
Chargenbezeichnung
Code du lot
Lotto Prodotto

CE

Legal manufacturer
Hersteller
Fabbricante

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ENGLISH

C REACTIVE PROTEIN

C-Reactive Protein (CRP) is an acute-phase protein that is produced mainly by the liver and increases in concentration as a result of injury, infection and inflammation. Although it is not a specific marker for a particular disorder, CRP is currently used as a generic diagnostic indicator of infection and inflammation and to monitor patients' reactions to treatment and post-operative recovery. CRP levels in the blood are high during bacterial infections and low during viral infections. For this reason, the dosage of C-reactive protein can be a useful tool to define the origin of an inflammatory state. Monitoring CRP levels over time helps to define the effectiveness of a therapy and evaluate a patient's healing process. CRP levels in the blood reach their peak concentration between 24 and 48 hours after the onset of the first symptoms of infection/inflammation, and then begin to decrease as the infection clears/inflammation subsides. The C-Reactive Protein Test provides a semi-quantitative evaluation of blood CRP levels that helps with therapeutic diagnosis and monitoring.

PRINCIPLE OF THE TEST

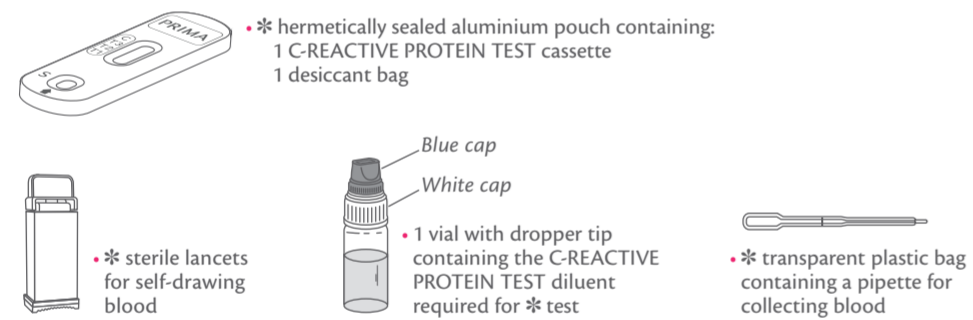
The C-REACTIVE PROTEIN TEST is a rapid immunochromatographic assay for the semi-quantitative detection of the C-Reactive Protein in human blood samples. It uses special gold-conjugated monoclonal antibodies integrated into the reactive strip.

REFERENCES

1. THOMPSON, D.; MILFORD-WARD, A.; WHICHER, J. T. „The value of acute phase protein measurements in clinical practice“. *Annals of clinical biochemistry*, 1992, 29.2: 123-131.
2. SHAW, A. C. „Serum C-reactive protein and neopterin concentrations in patients with viral or bacterial infection“. *Journal of clinical pathology*, 1991, 44.7: 596-599
3. C-reactive protein concentrations as a marker of inflammation or infection for interpreting biomarkers of micronutrient status. *Vitamin and Mineral Nutrition Information System*. Genf: World Health Organization; 2014 (https://www.who.int/nutrition/publications/micronutrients/indicators_c-reactive_protein/en/)
4. <https://www.mayoclinic.org/tests-procedures/c-reactive-protein-test/about/pac-20385228>

CONTENTS OF THE KIT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * antiseptic skin cleanser gauze
- * 1 instructions for use leaflet

* Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
* Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
* The desiccant packet must not be used. Dispose of it with household waste without opening it.

PRECAUTIONS

1. In vitro diagnostic device for individual use.
2. Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
3. Keep the Test out of the reach of children.
4. Do not use the Test after the expiry date, if the package is damaged or if the aluminium pouch is not sealed and intact.
5. Follow the procedure exactly, using only the specified quantities of blood and diluent.
6. Use the C-REACTIVE PROTEIN TEST cassette immediately after removal from the aluminium pouch.
7. To maintain the proper test functionality, store the components at temperatures between +4°C and +30°C. Do not freeze.
8. Use the Test and lancing device once only.
9. The test is for external use only. DO NOT SWALLOW.
10. Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
11. After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.
12. If you test another person, handle the blood samples as if containing a blood-borne infectious agent. Wear disposable gloves during use.

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