Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
support@primlabsa.ch
primlabsa.ch

Qarad EC-REP BV
Pas 257
2440 Geel
Belgium

800060_FU_11_7_1 07/2023

REF 800060

Registered trademarks.

INSTRUCTIONS FOR USE GEBRAUCHSANWEISUNG NOTICE D'UTILISATION ISTRUZIONI PER L'USO

0483

STREP A TEST

*Self-test for the detection of group A
β-hemolytic Streptococcus in throat swab*

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ENGLISH STREPTOCOCCUS A

Group A β-hemolytic Streptococcus (also known as *Streptococcus pyogenes*) is a Gram-positive bacterium that causes a variety of pathological conditions and complications, including pharyngitis (throat infections), skin infections (such as impetigo and erysipelas), acute glomerulonephritis, sepsis and rheumatic heart disease. Pharyngitis is commonly caused by viral infections and less often by bacterial infections, which require treatment with antibiotics.

STREP A TEST is an immunological test designed to specifically identify Streptococcus A bacteria in pharyngeal infections. It helps to understand quickly if a sore throat is caused by Streptococcus A or by other microorganisms that do not require treatment with antibiotics.

PRINCIPLE OF THE TEST

STREP A TEST is an immunochromatographic test that detects the presence of Streptococcus A specific antigens, using gold-conjugated antibodies integrated into the reactive strip.

REFERENCES

- Banerjee S, Ford C. "Rapid Tests for the Diagnosis of Group A Streptococcal Infection: A Review of Diagnostic Test Accuracy, Clinical Utility, Safety, and Cost-Effectiveness". Ottawa (ON): Canadian Agency for Drugs and Technologies in Health; 2018 May. CADTH Rapid Response Reports.
- Centers for Disease Control and Prevention. Adult appropriate antibiotic use summary. <http://www.cdc.gov/getsmart/community/materials-references/print-materials/hcp/adult-approp-summary.pdf> 2016.
- Llor C, Bjerrum L, Munck A, Cots JM, Hernández S, Moragas A; HAPPY AUDIT Investigators. "Access to point-of-care tests reduces the prescription of antibiotics among antibiotic-requesting subjects with respiratory tract infections." *Respir Care*. 2014 Dec.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

* hermetically sealed aluminium pouch containing:
1 STREP A TEST cassette
1 desiccant bag

* 1 dropper vial containing enough STREP A TEST - R1 solution to perform * Test R1 contains 20.7% Sodium Nitrite

* sterile swab
* sterile wooden tongue depressor

Danger H302: Harmful in case of ingestion. **Careful P264:** Wash carefully your hands after use. P270: Do not eat, drink or smoke while using this product.

* empty plastic test tube for sampling with dropper

* 1 dropper vial containing enough STREP A TEST - R2 solution to perform * Test R2 contains 0.55% Acetic Acid

* 1 instructions for use leaflet

* Do not open the sealed elements (test case, swab and tongue depressor) until just before performing the test. Take care to open as marked.
* The desiccant bag must not be used. Dispose of it with household waste without opening it.
* Material required but not supplied: a device to measure time (i.e. timer, watch).
* On the back of the test package there is a hole which can be used as a test tube holder.

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PRECAUTIONS

- Read these instructions for use carefully before performing the test. The Test is reliable if the instructions are followed correctly (reaction times, how the sample is collected and the extraction instructions shown).
- Keep the Test out of the reach of children.
- Not suitable for use on children under the age of 2 years.
- Do not use the Test after the expiry date or if the package has been damaged.
- Store the Test components at a temperature between +4 °C and +30 °C. Do not freeze.
- Use individual Tests once only.
- In vitro diagnostic device for individual use.
- The test is for external use only. Do not swallow solutions 1 and 2 and the other items in the box! If swallowed, seek medical advice immediately. Show the remaining parts of the box, the instructions for use and the package.
- If liquids come into contact with the eyes, skin or other mucosae, rinse immediately and thoroughly with water and contact a physician. Show the vials' labels.
- After using, dispose of all components according to local laws. Liquid wastes should not be disposed of down the drain.
- Do not open the sealed elements (test case, swab and tongue depressor) until just before performing the test. Take care to open as marked.
- Do not switch the caps of the reagent vials.
- All kit components are single-use, with the exception of the test package (which can be re-used as test tube holder) instructions for use and solution 1 and 2, which must be stored, closed with their caps, at a temperature between +4°C and +30°C.

MEDICAL DEVICES INSIDE THE KIT:
MEDIZINISCHE GERÄTE INNERHALB DES KITS:
DISPOSITIFS MÉDICAUX À L'INTÉRIEUR DU KIT:
DISPOSITIVI MEDICI INTERNI AL KIT:

STERILE R

Sterile swab
Steriler Tupfer
Tampone sterile
Tampons stériles

Capan Italia SpA
Via F. Perotti 10
25125 Brescia
Italy

STERILE EO

Sterile tongue depressor
Steriler Zungenspatel
Abaisse-langue stérile
Abbassalingua sterile

Applimed SA
Zl. Route de Pra de Pian 1,
CH-1618 Châtel-Saint-Denis
Switzerland

STERILE EO

Sterile tongue depressor
Steriler Zungenspatel
Abaisse-langue stérile
Abbassalingua sterile

Jiangsu Suyun Medical Materials Co., Ltd
No. 18, Jin Qiao Road Dapu Industrial Park
222002 Lianyungang, Jiangsu Province
People's Republic of China

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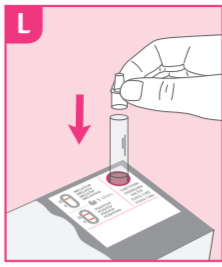
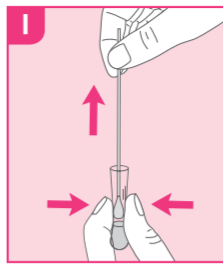
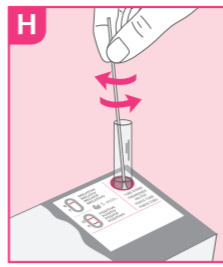
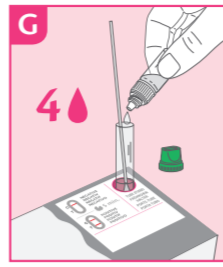
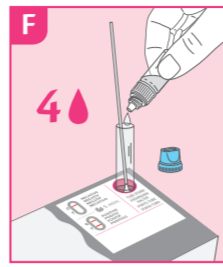
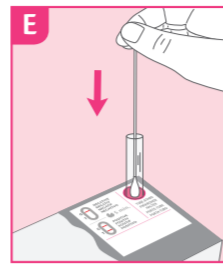
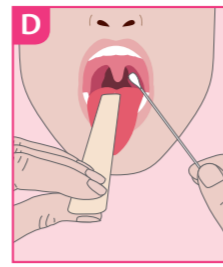
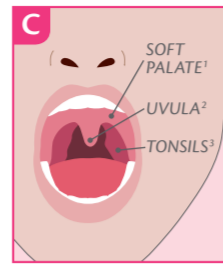
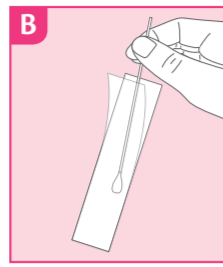
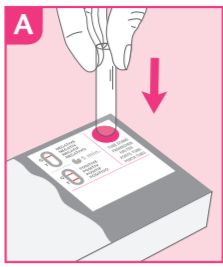
ENGLISH TEST PROCEDURE

WARNING: if purchasing a multipack, check that the sterile swab, the sterile tongue depressor, the test tube and the dropper have not been previously used (these disposable materials cannot be used twice).

- 1) Wash your hands with hot water and soap, rinse with clean water and dry.
2) Place the empty plastic test tube in the tube holder in the box.
3) Collecting the sample:
- Open the package containing the tongue depressor, leaving it inside the package and next to you.
- Open the package containing the sterile swab. Avoid touching the cotton tip and remove the swab using the plastic handle.
- Stand in front of a mirror, tilt your head backwards and open your mouth as much as possible.
- Pick up the tongue depressor with one hand and flatten the tongue.
- Use the other hand to place the swab close to the throat. Touch the back of the throat - the area around the tonsils...
4) After collecting sample, insert the swab's cotton tip into the supplied plastic test tube...
5) Open the vial STREP A TEST - R1 by unscrewing only the blue cap...
6) Next, open the vial STREP A TEST - R2 by unscrewing only the green cap...
7) Holding the swab's plastic handle, rotate the swab against the sides of the test tube about 10 times...
8) At the end of the incubation period, remove the test tube (with the swab and extraction fluid still in it) from the tube holder...
9) Add the supplied dropper to the plastic test tube.
10) Dispense 1 drop into the well in the test case.
11) Read the results after 5 minutes (some positive results can be seen after just 1 minute). Do not read after 10 minutes.

RESULTS INTERPRETATION

READ THE RESULT AFTER 5 MINUTES. DO NOT READ THE RESULT AFTER 10 MINUTES.

Diagram showing test results at 5 minutes. Includes sections for POSITIVE RESULT (two colored bands), NEGATIVE RESULT (one colored band), and NON VALID RESULT (no bands or line only under T).

F.A.Q. - QUESTIONS AND ANSWERS

HOW DOES THE STREP A TEST WORK? Group A beta-hemolytic Streptococcus could be involved in infections of the pharynx. For appropriate antibiotic treatment to be administered, the bacterium must be identified first. STREP A TEST can detect Streptococcus A antigens through the specific antibodies it contains.
WHEN CAN THE TEST BE USED? STREP A TEST can be carried out if there are symptoms such as pain when swallowing, sore throat, red and swollen tonsils...
CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may be incorrect if STREP A TEST is wet before performing the test or if the sample from the pharynx is not collected and prepared correctly...
WHAT IS THE PURPOSE OF THE CONTROL BAND? The appearance of this band means the test has worked correctly.
WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive, Streptococcus A has been detected in the sample from the pharynx and you must consult a physician.
WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the symptoms were not caused by the Streptococcus A or that the sample did not contain a concentration of bacteria that was high enough to be detected by the test.
IS THE RESULT RELIABLE IF READ AFTER 10 MINUTES? No. The test must be read 5 minutes after the procedure is completed and no later than 10 minutes from this time.
HOW ACCURATE IS THE STREP A TEST? The device specifically identifies Group A beta-hemolytic Streptococcus when its concentration in the extracted sample is equal to or higher than 10^4 CFU/mL (cut-off value).
WELCHEN ZWECK HAT DIE KONTROLLINIE? Wenn diese Linie erscheint, bedeutet dies, dass der Test korrekt funktioniert hat.

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Diagram showing test results at 5 minutes. Includes sections for POSITIVES RESULTAT (two colored lines), NEGATIVES RESULTAT (one colored line), and UNGÜLTIGES RESULTAT (no lines or line only under T).

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