

DRUG TEST

Self-test for the detection of 6 drugs in urine:
Morphine, Amphetamine, Cocaine, Marijuana,
Methamphetamine and Methadone



DETECTED SUBSTANCES

| TEST | Cut-off (ng/mL) | Time permanence in urine after exposure |
|---------------------------|-----------------|--|
| Amphetamine (AMP1000) | 1000 | 1-3 days for a single administration, up to 9 days in chronic users |
| Cocaine (COC 300) | 300 | 1-3 days for a single administration, up to 3 weeks in chronic users |
| Marijuana | 50 | single administration (smoking): 30 h on average, up to 4 days single administration (oral intake): up to 6 days occasional use (once or twice a week): up to 30 days regular consumption: up to 3 months |
| Methadone (MTD 300) | 300 | 1,5 to 3 days |
| Methamphetamine (MET 500) | 500 | 1 to 4 days for a single administration, up to 5 days in chronic users |
| Morphine (MOP 300) | 300 | 10-55 h for a single administration, up to 11 days in chronic users |

WHO ARE THE INTENDED USERS

DRUG TEST can be used by everyone who wants to check the presence of the above mentioned substances in urine.

TEST PRINCIPLE

DRUG TEST is an immunochromatographic assay that can be performed without the use of an instrument. The test utilizes monoclonal antibodies to selectively detect levels of specific drugs in urine.

TECH SPECS


| | CUT-OFF | SENSITIVITY | SPECIFICITY | OVERALL ACCURACY |
|-----|------------|-------------|-------------|------------------|
| AMP | 1000 ng/mL | 99,90% | 99,90% | 99,90% |
| MET | 500 ng/mL | 99,90% | 99,90% | 99,90% |
| COC | 300 ng/mL | 99,90% | 99,90% | 99,90% |
| MOP | 300 ng/mL | 99,90% | 99,90% | 99,90% |
| MTD | 300 ng/mL | 99,90% | 99,90% | 99,90% |
| THC | 50 ng/mL | 99,09% | 99,90% | 99,90% |

HOW TO USE IT

1) Open the cap, with the arrow pointing toward the urine specimen, dip the test panel vertically in the urine specimen for at least 10 to 15 seconds.

2) Replace the cap and leave the test panel on a non-absorbent flat surface.

3) Wait 5 minutes before reading the result.

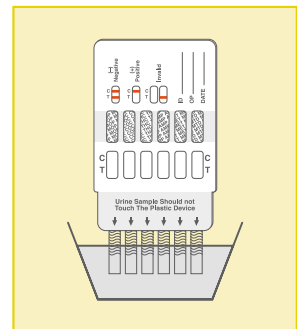

CONTENT: 1 sealed aluminium pouch containing: 1 test panel and 1 desiccant bag; 1 instructions for use leaflet.



VIDEO TUTORIAL

| REF | DESCRIPTION | NUMBER OF TESTS | SHELF LIFE |
|---------|-------------|-----------------|------------|
| 19806-1 | DRUG TEST | 1 TEST | 24 MONTHS |