Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND

EC REP Qarad EC-REP BV
Pas 257
2440 Geel - Belgium

support@primalabs.ch
primalabs.ch

800017_IFU_11_5.3 012024

REF 800017

Registered trademarks.

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

0483

HELICOBACTER PYLORI TEST

Self-test for the determination of anti Helicobacter pylori antibodies in whole blood samples

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EC REP

Huaian Trianda Medical Instruments Co., Ltd., No.106 East Songjiang Road, Huaiyin Economic&Technological Development Zone, 223002 - Huaian City, Jiangsu - P. R. China

Alcohol cleansing gauze
Alkohol-Reinigungsgaze
Gaze nettoyante à l'alcool
Garza detergente alcolica

EC REP

Phoenix Innovative Healthcare Manufacturers Pvt. Ltd., EL-209, Shil Mahape Road, Electronic Zone, MIDC, TTC Industrial Area, Mahape, Navi Mumbai 400 710 MH I India

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ENGLISH

HELICOBACTER PYLORI INFECTION AND SPECIFIC IgG

Helicobacter pylori is a bacterium whose ideal habitat is the human gastric mucosa. The infection is often asymptomatic, but it can sometimes cause gastritis and ulcers in the stomach or in the duodenum segment of the small intestine. The most common symptoms of these conditions are heartburn or pain in the upper part of the abdomen, especially if fasting or in the early morning, when the stomach is empty. Other associated symptoms include nausea, feeling full, reflux, loss of appetite, burping, weight loss and diarrhoea. Ulcers can sometimes bleed, causing anemia over long periods of time. In the long term, H. pylori infection is associated with an increase in the risk of gastric cancers. The simplest way to detect infection is by checking for the presence of specific antibodies (IgG) that are produced by the immune system after coming into contact with the bacterium and trying to eliminate it. The Helicobacter pylori Test can detect the presence of these antibodies in a few minutes.

PRINCIPLE OF THE TEST

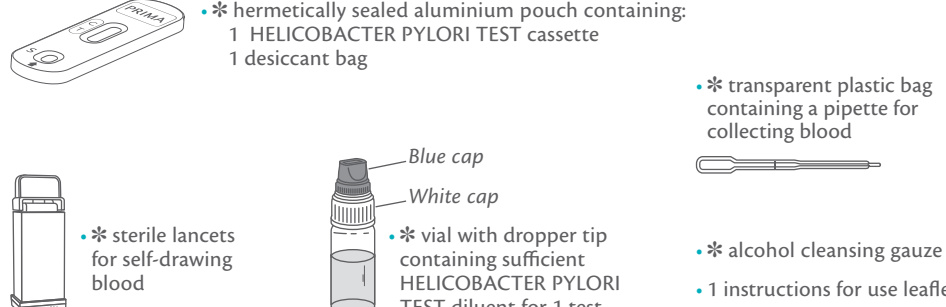
Helicobacter pylori Test is an immunochromatographic test that detects the presence of anti-Helicobacter pylori IgG antibodies in the blood, using special monoclonal antibodies conjugated with gold and integrated into the reactive strip.

REFERENCES

- Mati Moyat, Dominique Velin. Immune responses to Helicobacter pylori infection. World J. Gastroenterol. 2014 may 21; 20(19): 5583-5593IgE." Journal of Allergy and Clinical Immunology. 2014;133(2):589-91.
- Khean-Lee Goh, Wah-Kheong Chan, Seiji Shiota and Yoshio Yamaoka. Epidemiology of Helicobacter pylori Infection and Public Health Implications. Helicobacter, 2011 Sep;16(0 1):1-9.
- Kyichi Adachi, Tomoko Mishiro, Shino Tanaka and Yoshikazu Kinoshita. Analysis of negative result in serum anti-pylori IgG antibody test in cases with gastric mucosal atrophy. J. Clin Biochem Nutr.2016 Sep;59(2):145-148.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * hermetically sealed aluminium pouch containing:
1 HELICOBACTER PYLORI TEST cassette
1 desiccant bag
- * transparent plastic bag containing a pipette for collecting blood
- * sterile lancets for self-drawing blood
- * vial with dropper tip containing sufficient HELICOBACTER PYLORI TEST diluent for 1 test
- * alcohol cleansing gauze
- * 1 instructions for use leaflet

- * Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
- * Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
- * The desiccant packet must not be used. Dispose of it with household waste without opening it.

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