Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
support@primalabsa.ch
primalabsa.ch

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VITAMIN D TEST

Rapid self-test for the semi-quantitative detection
of vitamin D in human whole blood

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Huain Tianda Medical Instruments Co.,
Ltd., No.106 East Songjiang Road, Huaiyin,
Economic&Technological Development Zone,
223300 - Huain City, Jiangsu - P.R. China

Riomavix Sociedad Limitada
Calle de Almansa 55, 1D
Madrid 28039, Spain

Alcohol cleansing gauze
Alkohol-Reinigungsgaze
Gaze nettoyante à l'alcool
Garza detergente alcolica

Phoenix Innovative Healthcare Manufacturers Pvt. Ltd.,
E- 209, Shri Mahape Road, Electronic Zone, MIDC,
TTC Industrial Area, Mahape, Navi Mumbai
400 710 MH I India

Advena Ltd., Tower Business Centre,
2nd Flr, Tower Street, Swatar,
BKR 4013, Malta

SYMBOLS / SYMBOLE / SYMBOLES / SIMBOLOGIA

IVD In vitro diagnostic device In-Vitro-Diagnostikum Dispositif médical de diagnostic in vitro Dispositivo medico-diagnostico in vitro	TEMP Temperature limits Temperaturbegrenzung Limites de température Limiti di temperatura	REF List number Katalognummer Code produit Codice prodotto
I Read the instructions before use Beachten Sie die Gebrauchsanweisung Consultez le mode d'emploi Leggere attentamente le istruzioni per l'uso	DO NOT REUSE Do not reuse Nicht wiederverwenden Ne pas réutiliser Non riutilizzare	LOT Lot number Chargenbezeichnung Code du lot Lotto Prodotto
STERILE R Sterilized using irradiation Sterilisation durch Bestrahlung Méthode de stérilisation utilisant l'irradiation Sterilizzato mediante radiazioni	<n> Sufficient for <n> tests Ausreichend für <n> Tests Suffisant pour <n> tests Sufficiente per <n> test	CE CE marking CE-Kennzeichnung Marquage CE Marchio CE
EXP Expiry date (last day of the month) Verwendbar bis (letzter Tag des Monats) Utilisable jusqu'à (dernier jour du mois) Utilizzare entro (ultimo giorno del mese)	CONT Content Inhalt Contenu Contenuto	LEGAL Legal manufacturer Hersteller Fabricant Fabbricante
EU Authorised Representative in the European Community / Mandatant dans la Communauté européenne / Mandatario nella Comunità Europea		

ENGLISH

VITAMIN D

Vitamin D refers to a group of fat-soluble secosteroids responsible for increasing intestinal absorption of calcium, iron, magnesium, phosphate and zinc.

The blood concentration of 25-hydroxy Vitamin D (including D2 and D3) is considered the best indicator of Vitamin D status. Vitamin D deficiency is now recognized as a global epidemic. The health risks associated with Vitamin D deficiency are far more severe than previously thought. Vitamin deficiency has been linked to various serious diseases: osteoporosis, osteomalacia, multiple sclerosis, cardiovascular diseases, pregnancy complications, diabetes, depression, strokes, autoimmune diseases, flu, different cancers, infectious diseases, alzheimer, obesity and higher mortality etc. Therefore, now detecting Vitamin D level is considered as "Medically Necessary Screening Test", and maintaining sufficiently high levels of Vitamin D is necessary to improve bone health, health and well-being of the body.

PRINCIPLE OF THE TEST

The VITAMIN D TEST is an immunoassay based on the principle of competitive binding. During testing, the mixture migrates upward on the membrane chromatographically by capillary action. The membrane is pre-coated with 25 (OH) D antigens on the test line region of the strip.

LIMITATIONS

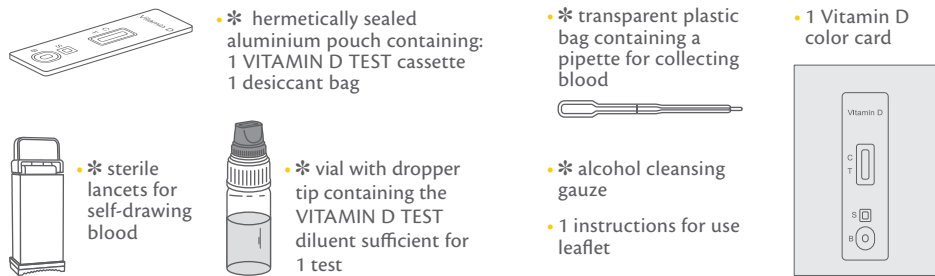
- The VITAMIN D TEST provides only a semi-quantitative analytical result. A secondary analytical method must be used to confirm the result.
- It is possible that technical or procedural errors, as well as other interfering substances in the whole blood specimen may cause erroneous results.
- As with all diagnostic tests, all results must be considered with other clinical information available to the physician.
- Other clinically available tests are required if questionable results are obtained.

BIBLIOGRAPHY

- Holick MF (March 2006). „High prevalence of vitamin D inadequacy and implications for health.“ Mayo Clinic Proceedings. 81 (3): 353–73.
- Eriksen EF, Glorup H (2002). Vitamin D deficiency and aging: implications for general health and osteoporosis. Biogerontology. 3 (1-2): 73-7.
- Grant WB, Holick MF (June 2005). Benefits and requirements of vitamin D for optimal health: a review. Alternative Medicine Review.10 (2): 94-111.
- Moyad MA. Vitamin D: a rapid review. DermatolNurs. 2009, 21:25-30.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

PRECAUTIONS

- For self-testing in vitro diagnostic use only.
- Do not eat, drink or smoke in the area where the specimens or kits are handled.
- Store in a dry place at 2-30°C (36-86°F), avoiding areas of excess moisture. If the foil packaging is damaged or has been opened, please do not use. The test is stable through the expiration date printed on the sealed pouch. The test must remain in the sealed pouch until use. DO NOT FREEZE. Do not use after the expiration date.
- This test kit is intended to be used as a preliminary test only and repeatedly abnormal results should be discussed with doctor or medical professional.
- Follow the indicated time strictly, using only the specified quantities of blood and diluent.
- Use the test only once. Do not dismantle and touch the test window of the test cassette.
- The kit must not be frozen or used after the expiration date printed on the package.
- Keep out of the reach of children.
- The test is for external use only. DO NOT SWALLOW.
- Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
- After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.
- If you test another person, handle the blood samples as if containing a blood-borne infectious agent. Wear disposable gloves during use.

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