TEST PROCEDURE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. Note: The use of warm water facilitates capillary blood collection as it induces vasodilation.
- 2) Lay out the material needed as follows: open the aluminium bag, take out only the test strip case and dispose of the desiccant packet. Open the plastic packet containing the pipette. Open the vial by unscrewing the white cap, making sure it does not fall to the ground. –**FIG. A**
- 3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it, then extract and discard the cap. –**FIG. B**
- 4) Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the fingertip, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip –**FIG. C**. The tip of the lancet automatically retracts after use. If the lancet does not work properly, discard it and use the second one supplied. If the second one is not required, it can be disposed of without special precautions.
- 5) Holding the hand downwards, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the fingertip to improve blood flow. –**FIG. D**
- 6) Take the pipette without pressing the bulb. Two sampling methods are suggested: –**FIG. E1**: hold the pipette horizontally **without pressing the bulb** place it in contact with the drop of blood, it will enter the pipette by capillarity. Move the pipette away when the black line is reached. If there is not enough blood, continue to massage the finger until the black line is reached. –**FIG. E2**: place the pipette on a clean, flat surface with the tip protruding from the surface, then place the drop of blood in contact with the pipette, it will enter by capillarity. If blood is not sufficient, continue massaging the finger until the blood has reached the black line. Avoid, as far as possible, to continuously move the tip of the pipette away from the finger in order to prevent the formation of air bubbles.
- 7) Deposit the blood collected with the pipette into the previously opened vial. Press the pipette bulb 2 or 3 times to ensure that all the blood has moved into the diluent solution. Replace the screw cap onto the vial and mix well for at least 10 seconds. –**FIG.F**
- 8) Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on). –**FIG.G**. Deposit **3 drops** into the well shown on the cassette (S). –**FIG.H**
- 9) **Wait 10 minutes** and then read the results as indicated.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 10 MINUTES

The intensity of the line colour does not have any importance for the interpretation of the test result.

NEGATIVE RESULT

A coloured line appears only under the C (control) sign.

This result means there are no anti-DGP antibodies in the blood or that their concentration is below the test's detection limit.

POSITIVE RESULTS

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The intensity of the T band may be lower than that of the C band. The test must therefore be considered positive regardless of the intensity of the T test band. This result shows the presence of anti-DGP antibodies in the blood and therefore probable celiac disease. Contact your GP to continue with diagnostic procedures.

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign.

In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE CELIAC TEST WORK? Gliadin is a protein component in gluten responsible for the immune reaction that causes intestinal inflammation in patients suffering from celiac disease. Recent studies of the diagnosis of celiac disease have shown that deamidated gliadin peptides in particular can trigger strong immune reactions. Antibodies directed against these peptides allow to diagnose celiac disease very accurately. The Celiac Test reveals the presence of IgA and IgG antibodies against deamidated gliadin peptides (anti-DGP). A positive result means that the concentration of these antibodies is approximately 20 U/ml (units/ml).

WHEN CAN THE TEST BE USED? The Celiac Test can be performed when symptoms such as diarrhoea, abdominal pain and swelling, weight loss, fatigue and/or skin reactions are present. The Test can be performed at any time of the day, on a full or empty stomach.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally spilt, if the amount of blood mixed with the diluent is not enough or the number of drops dispensed in the well is less than 3 or more than 4. The plastic pipette supplied allows users to be sure they have collected the right amount of blood.

IF I AM ON A GLUTEN-FREE DIET, CAN I TAKE THE TEST? No. In celiac disease patients on a gluten-free

diet the symptoms and the level of disease-associated antibodies decrease until they are no longer detectable. **HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT?** The test should be considered positive regardless of how dark the red/pink colour is of the test and control lines. The lines should be even and full. NB: if the control line is visible, the appearance of a white line at the T (Test) mark, associated with the absence of any red/pink colouring, should always be interpreted as a negative result.

IS THE RESULT RELIABLE IF READ AFTER 15 MINUTES? No. The test must be read 10 minutes after the procedure is completed (point 9) and no later than 15 minutes from this time.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the levels of class IgA and class IgG anti-DGP antibodies is approximately 20 U/ml. In this case you should consult your GP, who will decide how to proceed.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the levels of class IgA and class IgG anti-DGP antibodies are below the test's detection limit. If symptoms persist, seek medical advice.

HOW ACCURATE IS THE CELIAC TEST? The Test is very accurate. Assessment reports show a concordance rate above 90% (CI 95%: 82-98%) with reference methods.

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