TEST PROCEDURE

Before performing the test, collect a faeces sample following these instructions:

COLLECTING THE SAMPLE

- 1) Check the toilet seat is dry - contact with water or detergents may prevent the faeces-collecting sheet from adhering properly and interfere with the test result.
- 2) Use the supplied special stool-collecting sheet as described below:
 - a slowly detach the two adhesive ends of the faeces-collecting sheet in the direction pointed by the arrows printed on the sheet. Open the sheet completely;
 - b attach the two adhesive surfaces of the faeces-collecting sheet near the back of the toilet seat, making sure the sheet does not get wet; –**FIG. A**
 - c deposit the sample, carefully avoiding wetting the sheet with urine, and proceed with collection as described in the next steps in this procedure.
- 3) Unscrew the vial's red top and extract the faeces-collecting stick.
- 4) Dip the stick into the faeces in 3 different points of the sample. –**FIG. B**
- 5) Screw on the cap again, immersing the stick in the faeces extraction liquid, and shake the vial for at least 10 seconds. –**FIG. C**
- 6) Detach the adhesive parts of the faeces-collecting sheet at the same time from the toilet seat and let everything fall into the toilet. To make flushing easier, before flushing, wait a few seconds for the paper to soften. Flush

EXECUTION OF THE TEST

- 7) Open the aluminium bag, remove the test cassette only and place it on a horizontal surface. Discard the desiccant packet.
- 8) Break the end of the red cap, freeing the dropper portion. –**FIG. D**
- 9) Dispense 3 drops of the diluted faeces sample into the well shown on the test cassette. –**FIG. E**
Wait 1-2 seconds between dispensing one drop and the next. Do not move the device during the test performing to avoid invalid results.
- 10) Wait 10 minutes.

RESULTS INTERPRETATION

READ THE RESULT AFTER 10 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. This result means there is *Helicobacter pylori* antigen in the faeces and you must consult a physician.

NEGATIVE RESULTS

A coloured band appears only under the C (control) sign.

This result means that there is no *Helicobacter pylori* antigen in the faeces or that its concentration is below the test's detection limit (10 ng/ml).

NON VALID RESULT

If no bands appear or if there is only a line under the T (Test) sign, it means that it was not possible to read the result of the test, which must be considered as not valid. It is recommended to repeat the test with a new device and a new faeces sample.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE HELICOBACTER PYLORI ANTIGEN TEST WORK? The HELICOBACTER PYLORI ANTIGEN TEST was made to selectively identify the presence of *Helicobacter pylori* antigen in a faeces sample using a combination of specific antibodies. The lowest concentration of *Helicobacter pylori* antigen the test can detect is 10 ng/ml, although in some cases samples with *Helicobacter pylori* antigen concentration in the 7-10 ng/ml range may yield a positive result.

WHEN CAN THE TEST BE USED? The HELICOBACTER PYLORI ANTIGEN Test can be carried out at any time of the day, when you have a bowel movement, and there is no need to follow a specific diet.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally spilt, if the number of drops dispensed in the well is higher than 4 or less than 2.

If you are taking medication, please talk to your physician to check if the drugs you are taking alter the concentration of H. pylori antigen in the faeces.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CON-

TROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the test band.

IS THE RESULT RELIABLE IF READ AFTER 15 MINUTES? No. The test must be read exactly 10 minutes after the procedure is completed.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive, *Helicobacter pylori* antigen has been detected in the faeces and you must consult a physician, who will establish what to do next.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means the test has not detected any *Helicobacter pylori* antigen in the faeces or that the antigen's concentration is below the test's detection limit. Therefore negative results cannot guarantee there is no abnormal condition. If the result is negative and symptoms persist, talk to a physician.

HOW ACCURATE IS THE HELICOBACTER PYLORI ANTIGEN TEST? The Test is very accurate. Assessment reports show a concordance rate equal to 96% (95% CI: 92.2% - 99.4%) with reference methods. Despite the test's reliability, there may be false positive and false negative results.

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