Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
support@primalabsa.ch
primalabsa.ch

EC REP QbD RepS BV
Groenenborgerlaan 16
2610 Wilrijk - Belgium

Registered trademarks.

800018_IFU_11_8.0 11/2024

REF 800018-x

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

ALLERGY IgE TEST

Self-test for the determination of total class E Immunoglobulins (IgE)
in whole blood samples

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PRIMA
HOME TEST

ENGLISH ALLERGIES AND IgE

Class E immunoglobulins (IgE) are the class of antibodies that are most involved in allergic reactions. In normal conditions, IgE are present in the blood in very low quantity. However, when they come into contact with the substance that the body is allergic to, their concentration increases and they can trigger the cascade of reactions leading to the appearance of the main symptoms of an allergic reaction (such as skin erythema, itchiness, sneezing repeatedly, stinging and watery eyes and, in some cases, anaphylactic shock). This is why it is important to check if IgE levels are above normal, especially in individuals with a family history of allergies.

Test results showing that IgE levels are above average allow doctors to start looking for substances that cause an allergic reaction and to define a suitable therapy. The ALLERGY IgE TEST allows to check IgE levels in the blood and to determine sensitivity/ongoing allergic reaction.

PRINCIPLE OF THE TEST

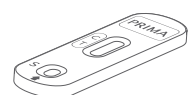
The ALLERGY IgE TEST is an immunochromatographic test that detects the presence of IgE antibodies in the blood in concentrations above average, using special monoclonal antibodies conjugated with gold and integrated into the reactive strip.

REFERENCES

- Martins T.B., Bandhauer M.E., Bunker A. M., Roberts W. L.: "New childhood and adult reference intervals for total IgE." *Journal of Allergy and Clinical Immunology*. 2014;133(2):589-91.
- Chang M.L., Cui C., Liu Y.H., Pei L.C., Shao B.: "Analysis of total immunoglobulin E and specific immunoglobulin E of 3,721 patients with allergic disease". *Biomed Rep*. 2015 Jul;3(4):573-577.
- I. Leonard Bernstein, MD; James T. Li, et AL: "Annals of allergy, asthma, & immunology." *March 2008; Vol 100, num 3, supplement 3.*


CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * hermetically sealed aluminium pouch containing:
1 ALLERGY IgE TEST cassette
1 desiccant bag

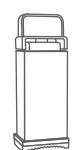- * transparent plastic bag containing a pipette for collecting blood

- * sterile lancets for self-drawing blood

- * vial with dropper tip containing sufficient ALLERGY IgE TEST diluent for 1 test

- * alcohol cleansing gauze
- * 1 instructions for use leaflet

- Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
- Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
- The desiccant packet must not be used. Dispose of it with household waste without opening it.

PRECAUTIONS

- Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
- Keep the Test out of the reach of children.
- Do not use the Test after the expiry date or if the package has been damaged.
- Follow the procedure exactly, using only the specified quantities of blood and diluent.
- Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
- Use the test and lancing device once only.
- The test is for external use only. DO NOT SWALLOW.
- In vitro diagnostic device for individual use.
- Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
- After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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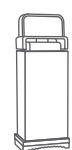
EC REP

Hualan Tianda Medical Instruments Co., Ltd., No.106 East Songjiang Road, Huaiyin, Economic&Technological Development Zone, 223300 - Hualan City, Jiangsu - P. R. China

Phoenix Innovative Healthcare Manufacturers Pvt. Ltd., EL-209, Shri Mahape Road, Electronic Zone, MIDC, TTC Industrial Area, Mahape, Navi Mumbai 400 710 MH I India

EC REP

Riomavix Sociedad Limitada
Calle de Almansa 55, 1D
Madrid 28039, Spain

EC REP

Advena Ltd., Tower Business Centre,
2nd Flr, TowerStreet, Swatar,
BKR 4013, Malta

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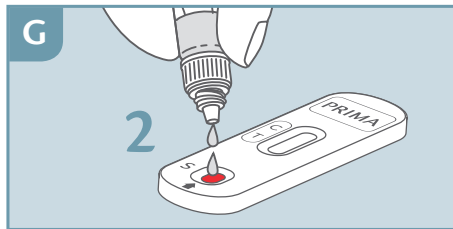
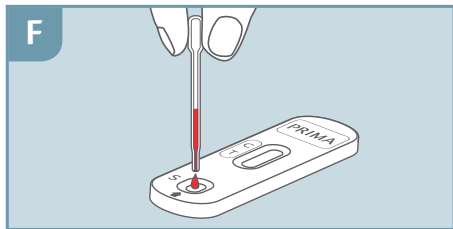
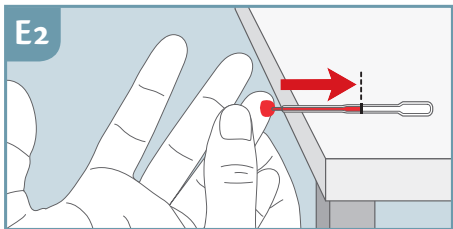
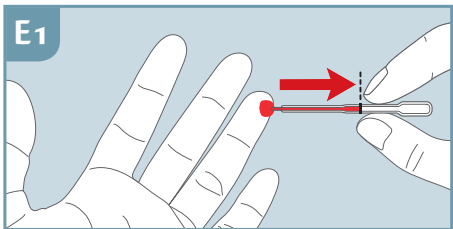
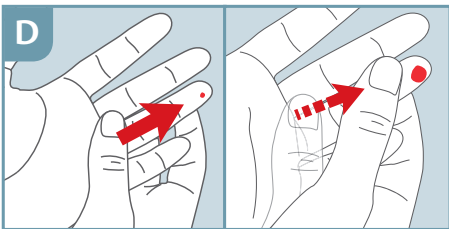
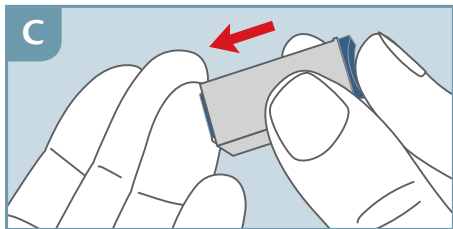
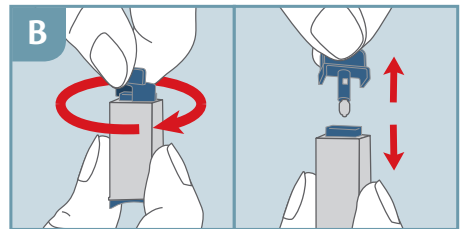
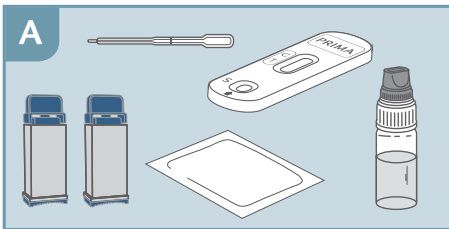
ENGLISH

TEST PROCEDURE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
Note: The use of warm water facilitates capillary blood collection as it induces vasodilation.
- 2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag. Open the plastic packet containing the pipette. –FIG. A
- 3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it. Extract and discard the released cap. –FIG. B
- 4) Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the phalanx, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip –FIG. C. The tip of the lancet automatically retracts after use. If the lancet does not work properly, discard it and use the second one supplied. If the second one is not required, it can be disposed of without special precautions.
- 5) Holding the hand down, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the phalanx to improve blood flow. –FIG. D
- 6) Take the pipette **without pressing the bulb**. Two sampling methods are suggested:
–FIG. E1: hold the pipette horizontally **without pressing the bulb** place it in contact with the drop of blood, it will enter the pipette by capillarity. Move the pipette away when the black line is reached. If there is not enough blood, continue to massage the finger until the black line is reached.
–FIG. E2: place the pipette on a clean, flat surface with the tip protruding from the shelf, then place the drop of blood in contact with the pipette, it will enter by capillarity. If blood is not sufficient, continue massaging the finger until the blood has reached the black line.
Avoid, as far as possible, to continuously move the tip of the pipette away from the finger in order to prevent the formation of air bubbles.
- 7) Place the blood collected with the pipette into the well indicated on the cassette (S) by pressing the pipette bulb. –FIG. F
- 8) Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on). Deposit **2 drops** into the well indicated on the cassette (S), wait for 5 seconds between the first and second drop. –FIG. G
- 9) **Wait 5 minutes** and read the results referring to the next section for result interpretation.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. This result means the levels of IgE in the blood are higher than normal and you should consult a physician.

NEGATIVE RESULTS

A coloured band appears only under the C (control) sign. This result means that the levels of IgE in the blood are normal or lower than the value used to calibrate the test (100 IU/mL).

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new blood sample.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE ALLERGY IgE TEST WORK? IgE antibodies are produced in high quantities by our body as a reaction to certain substances, defined allergens.

The ALLERGY IgE TEST specifically identifies human class IgE antibodies and reveals if the blood concentration of these antibodies is above 100 IU/mL. In some cases, samples with IgE concentration below this value (between 70 IU/mL and 99 IU/mL) can yield a positive result. The Test was calibrated on the 3rd International Standard NIBSC 11/234.

WHEN CAN THE TEST BE USED? The ALLERGY IgE Test is recommended in the presence of symptoms typical of allergic reactions, such as itching and/or reddened skin, watery eyes and sneezing repeatedly. The Test can be carried out at any time of the day. If you are taking medication, please talk to your physician to check that the drugs you are taking do not alter the levels of circulating IgE.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3. The pipette supplied allows users to be sure they have collected the right amount of blood.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the test band.

IS THE RESULT RELIABLE IF READ AFTER 5 MINUTES? No. The test must be read 5 minutes after the procedure is completed and no later than 6 minutes from this time.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the level of IgE class antibodies detected in the blood is higher than normal. You should consult a physician and show him/her the results of the test. The physician will decide which tests to perform or refer you to an allergy consultant.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the level of IgE class antibodies detected in the blood is lower than normal. If symptoms persist, consult your physician.

HOW ACCURATE IS THE ALLERGY IgE TEST? The Test is very accurate. Assessment reports show a concordance rate above 92% (CI 95%: 88.23 - 96.69%) with reference methods.

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