TEST PROCEDURE

Before performing the test, collect a faeces sample following these instructions:

COLLECTING THE SAMPLE

- 1) Check the toilet seat is dry - contact with water or detergents may prevent the faeces-collecting sheet from adhering properly and interfere with the test result.
- 2) Use the supplied special stool-collecting sheet as described below:
 - a slowly detach the two adhesive ends of the faeces-collecting sheet in the direction pointed by the arrows printed on the sheet. Open the sheet completely;
 - b attach the two adhesive surfaces of the faeces-collecting sheet near the back of the toilet seat, making sure the sheet does not get wet; –FIG. A
 - c deposit the sample, carefully avoiding wetting the sheet with urine, and proceed with collection as described in the next steps in this procedure.
- 3) Unscrew the vial's red top and extract the faeces-collecting stick.
- 4) Dip the stick into the faeces in 3 different points of the sample. –FIG. B
- 5) Screw on the cap again, immersing the stick in the faeces extraction liquid, and shake the vial for at least 10 seconds. –FIG. C
- 6) Detach the adhesive parts of the faeces-collecting sheet at the same time from the toilet seat and let everything fall into the toilet. To make flushing easier, before flushing, wait a few seconds for the paper to soften. Flush.

EXECUTION OF THE TEST

- 7) Open the aluminium bag, remove the test cassette only and place it on a horizontal surface. Discard the desiccant packet.
- 8) Break the end of the red cap, freeing the dropper portion. –FIG. D
- 9) Dispense 3 drops of the diluted faeces sample into the well shown on the test cassette. –FIG. E Wait 1-2 seconds between dispensing one drop and the next. Do not move the device during the test performing to avoid invalid results.
- 10) Wait 5 minutes.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (Test) and C (Control) signs. The T band may be less intense (lighter) than the C line. This result means there is blood in the faeces and you should consult a physician. You can obtain positive results even in the presence of haemoglobin concentration in the 36-40 ng/mL range.

NEGATIVE RESULTS

A coloured band appears only under the C (Control) sign. This result means that there is no blood in the faeces or that the concentration is below the test's detection limit (40 ng/mL).

NON VALID RESULT

If no bands appear or if there is only a line under the T (Test) sign, it means that it was not possible to read the result of the test, which must be considered as not valid. In this case it is not possible to interpret the result of the test, which must be considered not valid. It is recommended to repeat the test with a new BOWEL FOB TEST and new samples of faeces.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE BOWEL FOB TEST WORK? The BOWEL FOB TEST was made to selectively identify the presence of human haemoglobin in a stool sample using a combination of specific antibodies. The lowest concentration of human haemoglobin the test can detect is 40 ng/mL, although in some cases samples with haemoglobin concentration in the 36-40 ng/mL range may yield a positive result. **WHEN CAN THE TEST BE USED?** The BOWEL FOB TEST can be carried out at any time of the day, when you have a bowel movement, and there is no need to follow a specific diet. It should not be used while menstruating, as the result obtained may not be reliable. **CAN THE RESULT BE INCORRECT?** The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally spilt, if the number of drops dispensed in the well is higher than 4 or less than 2. **HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT?** The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be

full and homogeneous. The test is positive regardless of the intensity of the colour of the Test band. **IS THE RESULT RELIABLE IF READ AFTER 10 MINUTES?** No. The test must be read 5 minutes after the procedure is completed (point 10). **WHAT SHOULD I DO IF THE RESULT IS POSITIVE?** If the result is positive, blood has been detected in the faeces and you must consult a physician, who will establish what to do next. **WHAT SHOULD I DO IF THE RESULT IS NEGATIVE?** A negative result means the test has not detected any blood in the stool (some kinds of lesions bleed intermittently) or that the concentration of blood is below the test's detection limit. Therefore negative results cannot guarantee there is no abnormal condition. Only a doctor can make a diagnosis. **HOW ACCURATE IS THE BOWEL FOB TEST?** The Test is very accurate. Assessment reports show a concordance rate above 92.6% (95% CI: 88.4 - 96.8%) with reference methods. Despite the test's reliability, there may be false positive and false negative results.

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