Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
support@primalabs.ch
primalabs.ch

EC REP QbD RepS BV
Groenenborgerlaan 16
2610 Wilrijk - Belgium

800017_IFU_11_8.0 11/2024 **REF** 800017-x

Registered trademarks.

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

HELICOBACTER PYLORI TEST

Self-test for the determination of anti Helicobacter pylori antibodies in whole blood samples

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PRIMA
HOME TEST

ENGLISH HELICOBACTER PYLORI INFECTION AND SPECIFIC IgG

Helicobacter pylori is a bacterium whose ideal habitat is the human gastric mucosa. The infection is often asymptomatic, but it can sometimes cause gastritis and ulcers in the stomach or in the duodenum segment of the small intestine. The most common symptoms of these conditions are heartburn or pain in the upper part of the abdomen, especially if fasting or in the early morning, when the stomach is empty. Other associated symptoms include nausea, feeling full, reflux, loss of appetite, burping, weight loss and diarrhoea. Ulcers can sometimes bleed, causing anemia over long periods of time. In the long term, H. pylori infection is associated with an increase in the risk of gastric cancers. The simplest way to detect infection is by checking for the presence of specific antibodies (IgG) that are produced by the immune system after coming into contact with the bacterium and trying to eliminate it. The Helicobacter pylori Test can detect the presence of these antibodies in a few minutes.

PRINCIPLE OF THE TEST

Helicobacter pylori Test is an immunochromatographic test that detects the presence of anti-Helicobacter pylori IgG antibodies in the blood, using special monoclonal antibodies conjugated with gold and integrated into the reactive strip.

REFERENCES

- Mati Moyat, Dominique Velin. Immune responses to Helicobacter pylori infection. World J. Gastroenterol. 2014 may 21; 20(19): 5583-5593IgE." Journal of Allergy and Clinical Immunology. 2014;133(2):589-91.
- Khean-Lee Goh, Wah-Kheong Chan, Seiji Shiota and Yoshio Yamaoka. Epidemiology of Helicobacter pylori Infection and Public Health Implications. Helicobacter, 2011 Sep;16(0 1):1-9.
- Kyichi Adachi, Tomoko Mishi, Shino Tanaka and Yoshikazu Kinoshita. Analysis of negative result in serum anti-pylori IgG antibody test in cases with gastric mucosal atrophy. J. Clin Biochem Nutr.2016 Sep;59(2):145-148.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the “content” section on the external box.

- * hermetically sealed aluminium pouch containing:
1 HELICOBACTER PYLORI TEST cassette
1 desiccant bag
- * transparent plastic bag containing a pipette for collecting blood
- * sterile lancets for self-drawing blood
- * Blue cap
- * White cap
- * vial with dropper tip containing sufficient HELICOBACTER PYLORI TEST diluent for 1 test
- * alcohol cleansing gauze
- * 1 instructions for use leaflet

- Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
- Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
- The desiccant packet must not be used. Dispose of it with household waste without opening it.

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CE 0197

Huaian Tianda Medical Instruments Co., Ltd., No.106 East Songjiang Road, Huaiyin, Economic&Technological Development Zone, 223300 - Huaian City, Jiangsu - P. R. China

EC REP

Riomavix Sociedad Limitada
Calle de Almansa 65, 10
Madrid 28039, Spain

Alcohol cleansing gauze
Alkohol-Reinigungsgaze
Gaze nettoyante à l'alcool
Garza detergente alcolica

CE

Phoenix Innovative Healthcare Manufacturers Pvt. Ltd. EL-209, Shil Mahape Road, Electronic Zone, MIDC, TTC Industrial Area, Mahape, Navi Mumbai 400 710 MH I India

EC REP

Advena Ltd., Tower Business Centre, 2nd Flr, Tower Street, Swatar, BKR 4013, Malta

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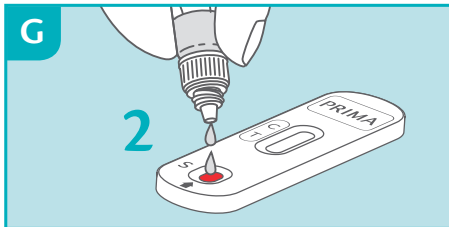
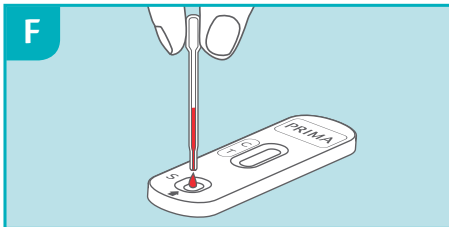
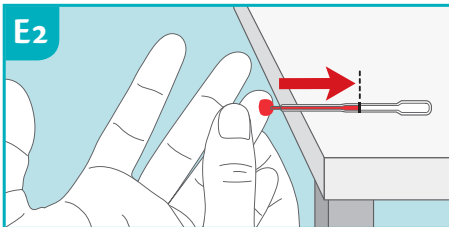
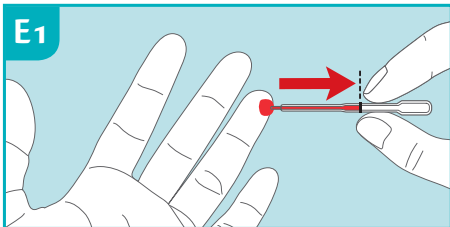
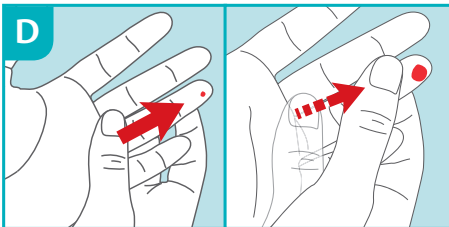
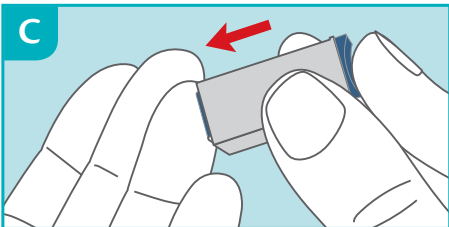
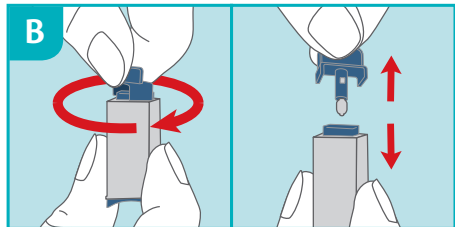
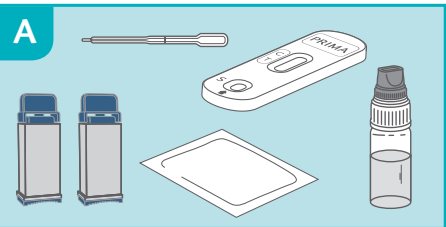
TEST PROCEDURE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
Note: The use of warm water facilitates capillary blood collection as it induces vasodilation.
- 2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag. Open the plastic packet containing the pipette. –**FIG. A**
- 3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it. Extract and discard the released cap. –**FIG. B**
- 4) Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the phalanx, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip –**FIG. C** The tip of the lancet automatically retracts after use. If the lancet does not work properly, discard it and use the second one supplied. If the second one is not required, it can be disposed of without special precautions.
- 5) Holding the hand down, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the phalanx to improve blood flow. –**FIG. D**
- 6) Take the pipette **without pressing the bulb**. Two sampling methods are suggested:
–**FIG. E1:** hold the pipette horizontally **without pressing the bulb** place it in contact with the drop of blood, it will enter the pipette by capillarity. Move the pipette away when the black line is reached. If there is not enough blood, continue to massage the finger until the black line is reached.
–**FIG. E2:** place the pipette on a clean, flat surface with the tip protruding from the shelf, then place the drop of blood in contact with the pipette, it will enter by capillarity. If blood is not sufficient, continue massaging the finger until the blood has reached the black line.
Avoid, as far as possible, to continuously move the tip of the pipette away from the finger in order to prevent the formation of air bubbles.
- 7) Place the blood collected with the pipette into the well indicated on the cassette (S) by pressing the pipette bulb. –**FIG. F**
- 8) Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on). Deposit **2 drops** into the well indicated on the cassette (S), wait for 5 seconds between the first and second drop. –**FIG. G**
- 9) **Wait 5 minutes** and read the results referring to the next section for result interpretation.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line.
This result means there are anti-Hp antibodies in the blood and you must consult a physician.

NEGATIVE RESULTS

A coloured band appears only under the C (control) sign.
This result means that the presence of anti-Hp antibodies cannot be detected in the blood.

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign.
In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new blood sample.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE HELICOBACTER PYLORI TEST WORK? IgG antibodies are produced in high quantities by our body in response to infections. The Helicobacter pylori Test identifies class IgG antibodies specific for Helicobacter pylori and reveals if the concentration in the blood of these antibodies is 15 AU/mL. In some cases, samples with IgG concentrations between 10 AU/mL and 14.99 AU/mL can yield positive results.

WHEN CAN THE TEST BE USED? HELICOBACTER PYLORI TEST is recommended when the typical symptoms of Helicobacter pylori gastric infection are present, such as stomach pain, heartburn, nausea, feeling full, burping, loss of appetite, diarrhoea and weight loss. The Test can be carried out at any time of the day. If you are taking medication, please talk to your physician to check that the drugs you are taking do not alter the levels of circulating IgG.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally spilt, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3. The pipette supplied allows users to be sure they have collected the right amount of blood.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the

result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the test band.

IS THE RESULT RELIABLE IF READ AFTER 5 MINUTES? No. The test must be read 5 minutes after the procedure is completed.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the level of anti-Helicobacter pylori specific antibodies belonging to the IgG class is equal to or higher than 15 AU/mL. The test does not discriminate between ongoing infection or past infections, so consult a physician and show him/her the result of the test. Your physician will decide which other tests you should perform and advise you accordingly.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the level of anti-Helicobacter pylori specific antibodies belonging to the IgG class is lower than standard. If symptoms persist, consult your physician.

HOW ACCURATE IS THE HELICOBACTER PYLORI TEST? The Test is very accurate. Assessment reports show a concordance rate above 87% (CI 95%: 80.4 – 93.6%) with reference methods.

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