- TEST PROCEDURE**
- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
Note: The use of warm water facilitates capillary blood collection as it induces vasodilation.
 - 2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag. Open the plastic packet containing the pipette. –**FIG. A**
 - 3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it. Extract and discard the released cap. –**FIG. B**
 - 4) Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the phalanx, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip –**FIG. C** The tip of the lancet automatically retracts after use. If the lancet does not work properly, discard it and use the second one supplied. If the second one is not required, it can be disposed of without special precautions.
 - 5) Holding the hand down, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the phalanx to improve blood flow. –**FIG. D**
 - 6) Take the pipette **without pressing the bulb**. Two sampling methods are suggested:
–**FIG. E1:** hold the pipette horizontally **without pressing the bulb** place it in contact with the drop of blood, it will enter the pipette by capillarity. Move the pipette away when the black line is reached. If there is not enough blood, continue to massage the finger until the black line is reached.
–**FIG. E2:** place the pipette on a clean, flat surface with the tip protruding from the shelf, then place the drop of blood in contact with the pipette, it will enter by capillarity. If blood is not sufficient, continue massaging the finger until the blood has reached the black line.
Avoid, as far as possible, to continuously move the tip of the pipette away from the finger in order to prevent the formation of air bubbles.
 - 7) Place the blood collected with the pipette into the well indicated on the cassette (S) by pressing the pipette bulb. –**FIG. F**
 - 8) Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on). Deposit **2 drops** into the well indicated on the cassette (S), wait for 5 seconds between the first and second drop. –**FIG. G**
 - 9) **Wait 5 minutes** and read the results referring to the next section for result interpretation.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line.
This result means the levels of IgE in the blood are higher than normal and you should consult a physician.

NEGATIVE RESULTS

A coloured band appears only under the C (control) sign.
This result means that the levels of IgE in the blood are normal or lower than the value used to calibrate the test (100 IU/mL).

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign.
In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new blood sample.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE ALLERGY IgE TEST WORK? IgE antibodies are produced in high quantities by our body as a reaction to certain substances, defined allergens.

The ALLERGY IgE Test specifically identifies human class IgE antibodies and reveals if the blood concentration of these antibodies is above 100 IU/mL. In some cases, samples with IgE concentration below this value (between 70 IU/mL and 99 IU/mL) can yield a positive result. The Test was calibrated on the 3rd International Standard NIBSC 11/234.

WHEN CAN THE TEST BE USED? The ALLERGY IgE Test is recommended in the presence of symptoms typical of allergic reactions, such as itching and/or reddened skin, watery eyes and sneezing repeatedly. The Test can be carried out at any time of the day. If you are taking medication, please talk to your physician to check that the drugs you are taking do not alter the levels of circulating IgE.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3. The pipette supplied allows users to be sure they have collected the right amount of blood.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the test band.

IS THE RESULT RELIABLE IF READ AFTER 5 MINUTES? No. The test must be read 5 minutes after the procedure is completed and no later than 6 minutes from this time.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the level of IgE class antibodies detected in the blood is higher than normal. You should consult a physician and show him/her the results of the test. The physician will decide which other tests to perform or refer you to an allergy consultant.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the level of IgE class antibodies detected in the blood is lower than normal. If symptoms persist, consult your physician.

HOW ACCURATE IS THE ALLERGY IgE TEST? The Test is very accurate. Assessment reports show a concordance rate above 92% (CI 95%: 88.23 - 96.69%) with reference methods.

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