WHEN TO PERFORM THE TEST?

To understand when to perform the test you must know how long your cycle lasts. The length of a cycle is from the first day your period begins and the day the next period begins. Use the table 1 to establish the day in which you should start testing. The first day of your period is shown as day 1. If your period is irregular, consider the duration of the shortest cycle in the last 6 months. Refer to the table to establish the day in which you should start testing. If your cycle is shorter than 21 days or longer than 40, please talk to your physician.

TAB. 1		1	2	3	4	5	6
CYCLE DURATION BEGIN OF THE TEST	21-22 days	7	8	9	10	11	12
	23-24 days	8	9	10	11	12	13
	25 days	9	10	11	12	13	14
	26 days	10	11	12	13	14	15
CYCLE DURATION BEGIN OF THE TEST	27 days	11	12	13	14	15	16
	28 days	12	13	14	15	16	17
	29 days	13	14	15	16	17	18
	30 days	14	15	16	17	18	19
CYCLE DURATION BEGIN OF THE TEST	31 days	15	16	17	18	19	20
	32 days	16	17	18	19	20	21
	33 days	17	18	19	20	21	22
	34 days	18	19	20	21	22	23
CYCLE DURATION BEGIN OF THE TEST	35 days	19	20	21	22	23	24
	36 days	20	21	22	23	24	25
	37 days	21	22	23	24	25	26
	38 days	22	23	24	25	26	27
CYCLE DURATION BEGIN OF THE TEST	39 days	23	24	25	26	27	28
	40 days	24	25	26	27	28	29
	41 days	25	26	27	28	29	30
	42 days	26	27	28	29	30	31

EXAMPLE: the average duration of your cycle is 28 days. Your last period started on the 3rd of the month. The table 1 shows that you should count 11 days from the 3rd day of the month (included), so the first test should be done on the 14th day of that month. –FIG. A

TEST PROCEDURE

- 1) Open the protective bag and take out the Test device. Dispose of the desiccant packet contained in the aluminium bag.
- 2) Remove the protective cap - this exposes the absorbing tip. –FIG. B
Rotate the device so that the results window faces up and the absorbing tip faces down.
- 3) Place the absorbing tip right under the flow of urine for at least 10 seconds. –FIG. C1
WARNING: Ensure no drops of urine splash onto the reading window. Alternatively (if easier), urine can be collected in a clean, dry, residue-free container (not supplied) and the absorbing tip can be dipped into the urine for 10 seconds –FIG. C2
- 4) Place the protective cap back onto the absorbing tip. –FIG. D
- 5) Put the test stick onto a dry, flat surface with the results window facing up. –FIG. E

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the results window in the T (Test) and C (Control) regions. The T (Test) band may be less intense (lighter) than the C (Control) line. This means that the concentration of the luteinizing hormone (LH) in the urine is equal to or higher than 25 mIU/ml. In this case you are reaching the ovulation peak and the chances of conceiving have increased.

NEGATIVE RESULT

A coloured line appears only in the C (Control) region. This means that the concentration of luteinizing hormone (LH) in the urine is below 25 mIU/ml and that you have not reach the ovulation peak. Daily testing is recommended.

NON VALID RESULT

No line appears in the C (Control) region and it is impossible to interpret the result of the test, which must be considered not valid. You should repeat the test with a new OVULATION LH TEST and a new urine sample. The most frequent causes of incorrect use are:
a) the absorbing tip was immersed in the urine for too long or not long enough;
b) drops of urine splashed onto the results window while the tip was under the urine flow.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE OVULATION LH TEST WORK? The device specifically identifies the human luteinizing hormone (LH) and reveals if the concentration of that hormone in the urine is equal to or above 25 mIU/ml. In some cases, samples with LH concentration below this value (between 22 mIU/ml and 24.99 mIU/ml) can yield a positive result. The Test was calibrated on the 3rd International Standard NIBSC 81/535.
AT WHAT TIME OF THE DAY SHOULD THE TEST BE CARRIED OUT? The test can be carried out at any time of the day, but some women have the best results with urine collected in the afternoon. If necessary, continue testing every day at the same time until you find the LH peak. Do not use with early morning urine.
CAN THE RESULTS BE INCORRECT? The correct results are obtained if the instructions are followed correctly. However, there may be incorrect results if moisture ends up on the device before the test is carried out. Additionally, in rare cases, the results may be erroneously positive or negative in women who are suffering from hormonal metabolic disorders. In the case of unexpected or contradictory results, you should talk to your gynaecologist, because some health conditions may alter the test's results.

IF THE ABSORBING TIP IS LEFT IN THE URINE FOR MORE THAN 10 SECONDS, IS THE RESULT CORRECT? Yes, a few seconds more do not interfere with the result.
IF THE ABSORBING TIP IS LEFT IN THE URINE FOR LESS THAN 10 SECONDS, IS THE RESULT CORRECT? No, because the volume of the absorbed sample may not be enough.
WHAT SHOULD I DO IF THE RESULT IS POSITIVE? Increase the frequency of sexual intercourse because there is a greater chance of conceiving.
WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? If the result is negative, the levels of LH in the urine are below 25 mIU/ml, which means the chances of conceiving are low. You should repeat the test every day to identify the increase in hormone concentration.
CAN THE TEST BE USED AS BIRTH CONTROL? No, the test has not been designed for that purpose and it must not be used for that purpose.
DOES GOING OFF ORAL CONTRACEPTIVES (THE PILL) INTERFERE WITH THE RESULTS? For some time after you stopped taking oral contraceptives, your periods may be irregular, which could interfere with the results.
HOW ACCURATE IS THE OVULATION LH TEST? Assessment reports show a concordance rate above 98% (CI 95%: 94 - 100%) with reference methods.

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