Test your health

PRIMA LAB SA
Viale Serfontana, 10
CH-6834 Morbio Inferiore - SWITZERLAND
support@primlabsa.ch
primlabsa.ch

19806_IFU_11_9.0 03/2026 REF 19806-x

Registered trademarks.

INSTRUCTIONS FOR USE GEBRAUCHSANWEISUNG NOTICE D'UTILISATION ISTRUZIONI PER L'USO

DRUG TEST

Self-Test for the detection of 6 drugs in urine: Morphine, Amphetamine, Cocaine, Marijuana, Methamphetamine and Methadone

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ENGLISH MULTIDRUG RAPID TEST

A rapid self-test for the simultaneous, qualitative detection of multiple drugs and drug metabolites in human urine. The DRUG TEST is a rapid chromatographic immunoassay for the qualitative detection of multiple drugs and drug metabolites in urine. Their permanence in urine and their detective levels are expressed in the following table:

TEST	Cut-off (ng/mL)	Time permanence in urine after exposure*
Amphetamine (AMP1000)		
d-Amphetamine	1.000	1-3 days for a single administration, up to 9 days in chronic users
Cocaine (COC 300)		
Benzoylecgonine	300	1-3 days for a single administration, up to 3 weeks in chronic users
Marijuana		
11-nor- Δ^9 -THC-9 COOH (THC-COOH)	50	single administration (smoking): 30 h on average, up to 4 days single administration (oral intake): up to 6 days occasional use (once or twice a week): up to 30 days regular consumption: up to 3 months
Methadone (MTD 300)		
Methadone	300	1,5 to 3 days
Methamphetamine (MET 500)		
d-Methamphetamine	500	1 to 4 days for a single administration, up to 5 days in chronic users
Morphine (MOP 300)		
Morphine	300	10-55 h for a single administration, up to 11 days in chronic users
6-acetylmorphine	300	2 to 4,5 h for a single administration, up to 35 h in chronic users

* according to SCDAT guidelines for Drugs of Abuse Testing, Vers EN 2021-03-25 (corrected 2022-12-6)

This assay provides only a preliminary analytical test result. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are indicated.

PRINCIPLE OF THE TEST

The DRUG TEST Panel is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes monoclonal antibodies to selectively detect elevated levels of specific drugs in urine. The speed and sensitivity of the immunochemical techniques have made themselves the most relevant methods for drugs of abuse screening. During testing, a urine specimen migrates upward by capillary action. A drug, if present in the urine specimen below its cut-off concentration, will not saturate the binding sites of its specific antibody. The antibody will then react with the drug-protein conjugate and a visible colored line will show up in the test region of the specific drug dipstick. The presence of drug above the cut-off concentration will saturate all the binding sites of the antibody. Therefore, the colored line will not form in the test region. A drug-positive urine specimen will not generate a colored line in the specific test region of the dipstick because of drug competition, while a drug-negative urine specimen will generate a line in the test region because of the absence of drug competition. To serve as a procedural control, a colored line will always appear at the control region, indicating that proper volume of specimen has been added and membrane wicking has occurred.

KIT CONTENT

Materials provided
1 Test cassette
1 Instructions for Use

Materials needed but not provided
Timer
Urine collection cup

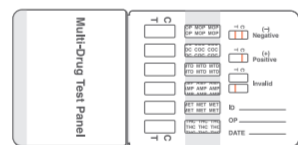
Image for illustration purposes only

LIMITATIONS

- 1) The DRUG TEST Panel provides only a qualitative, preliminary analytical result.
- 2) There is a possibility that technical or procedural errors, as well as interfering substances in the urine specimen may cause erroneous results.
- 3) A positive result does not indicate level or intoxication or concentration in urine.
- 4) A negative result may not necessarily indicate drug-free urine. Negative results can be obtained when drug is present but below the cut-off level of the test.
- 5) This test does not distinguish between drugs of abuse and certain medications.

PRECAUTIONS

1. The assay is designed for use with human urine only. Fresh urine specimens should be used, and the urine does not require any special handling or pretreatment.
2. Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected.
3. Keep out of the reach of children.
4. The test Panel should remain in the sealed pouch until use.
5. Do not use after the expiry date printed on the label and on the protective pouch.
6. Make sure you correctly follow the "time" instructions when carrying out the test and observing the results.
7. Do not freeze. Use the test kit at temperatures between 15-30 °C.
8. After use, all components can be disposed according to local regulations; all specimen should be considered potentially hazardous and handled in the same manner as an infectious agent.
9. The test kits should be kept away from direct sunlight, moisture and heat.
10. In case of difficulties in color identification (such as Daltonism), ask for help in test reading.
11. External use only. DO NOT SWALLOW.

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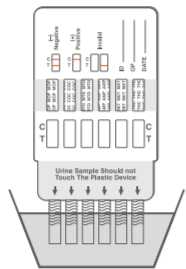
SPECIMEN HANDLING AND STORAGE

The urine specimen should be collected in a clean and dry container. Adulterants, such as bleach and/or alcohol, in urine specimens, may produce erroneous results regardless of the analytical method used. If adulteration is suspected, the test should be repeated with another urine specimen. Urine collected at any time of the day may be used. Urine specimens exhibiting visible precipitates should be centrifuged, filtered, or allowed to settle to obtain a clear specimen for testing. Urine specimens may be stored at 2–8 °C for up to 48 hours prior to testing. For prolonged storage, specimens may be frozen and stored below –20 °C. Frozen specimens should be thawed and mixed well before testing.

TEST PROCEDURE

Allow the test and the urine specimen, and/or controls to reach room temperature (15–30 °C) prior to testing.

- Bring the pouch to room temperature before opening it. Remove the test panel from the sealed pouch and use it within one hour.
- Remove the cap.
- With the arrow pointing toward the urine specimen, immerse the test panel vertically in the urine specimen for at least 10 to 15 seconds. Immerse the dipstick to at least the level of the wavy lines, but not above the arrow on the test panel.
- Replace the cap and place the test panel on a non-absorbent flat surface.
- Start the timer and wait for the colored line(s) to appear.

**RESULTS INTERPRETATION**

RESULT SHOULD BE READ AT 5 MINUTES.

DO NOT INTERPRET THE RESULT AFTER 10 MINUTES.

The shade of the colored lines in the Test (T) region may vary. The result should be considered negative whenever there is even a faint line. Please refer to the illustration

**NEGATIVE**

A colored line appears in the Control (C) region and colored lines appear in the Test (T) region. This negative result means that the concentrations in the urine sample are below the designated cut-off levels for a particular drug tested.

POSITIVE

A colored line appears in the Control (C) region and no line appears in the Test (T) region. The positive result means that the drug concentration in the urine sample is greater than the designated cut-off for a specific drug.

NON VALID

No line appears in the Control (C) region. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for Control line failure. Read the directions again and repeat the test with a new test card. If the result is still invalid, contact your manufacturer.

Amphetamine (AMP) Amphetamine is a potent central nervous system (CNS) stimulant that is used in the treatment of attention deficit hyperactivity disorder (ADHD), narcolepsy, and obesity but it is also available on the illicit market. Amphetamines are chemically related to the human body's natural catecholamines: epinephrine and norepinephrine. Acute higher doses lead to enhanced stimulation of the central nervous system (CNS) and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to amphetamines include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, and psychotic behavior. The effects of amphetamines generally last 2-4 hours following use and the drug has a half-life of 4-24 hours in the body. About 30% of amphetamines are excreted in the urine in unchanged form, with the remainder as hydroxylated and deaminated derivatives. The DRUG TEST Panel yields a positive result when the concentration of amphetamines in urine exceeds the detection level of 1000 ng/mL.

Cocaine (COC) Cocaine is a potent central nervous system stimulant and a local anesthetic. Initially, it brings about extreme energy and restlessness while gradually resulting in tremors, over-sensitivity and spasms. In large amounts, cocaine causes fever, unresponsiveness, difficulty in breathing and unconsciousness. Cocaine is often self-administered by nasal inhalation, intravenous injection and free-base smoking. It is excreted in the urine in a short time primarily as benzoylecgonine. Benzoylecgonine, a major metabolite of cocaine, has a longer biological half-life (5-8 hours) than cocaine (0.5-1.5 hours), and can generally be detected for 24-72 hours after cocaine exposure. The DRUG TEST Panel yields a positive result when the concentration of benzoylecgonine in urine exceeds the detection level of 300 ng/mL.

Marijuana (THC) THC (Δ9-tetrahydrocannabinol) is the primary active ingredient in cannabis (marijuana). When smoked or orally administered, THC produces euphoric effects. Users have impaired short-term memory and slowed learning. They may also experience transient episodes of confusion and anxiety. Long-term, relatively heavy use may be associated with behavioral disorders. The peak effect of marijuana administered by smoking occurs in 20-30 minutes and the duration is 90-120 minutes after one cigarette. Elevated levels of urinary metabolites are found within hours of exposure and remain detectable up to 4 days after smoking. The main metabolite excreted in the urine is 11-nor-Δ9-tetrahydrocannabinol-9-carboxylic acid (THC-COOH). The DRUG TEST Panel yields a positive result when the concentration of THC-COOH in urine exceeds the detection level of 50 ng/mL.

Methadone (MTD) Methadone is a narcotic analgesic prescribed for the management of moderate to severe pain and for the treatment of opiate dependence (Heroin, Vicodin, Percocet, Morphine). The pharmacology of oral methadone is very different from IV methadone. Oral methadone is partially stored in the liver for later use. IV methadone acts more like heroin. In most states you must go to a pain clinic or a methadone maintenance clinic to be prescribed methadone. Methadone is a long acting pain reliever producing effects that last from 3-5 days. Ideally, methadone frees the client from the pressures of obtaining illicit heroin, from the dangers of injection, and from the emotional roller coaster that most opiates produce. Methadone, if taken for long periods and at large doses, can lead to a very long withdrawal period. The withdrawals from methadone are more prolonged and troublesome than those provoked by heroin cessation, yet the substitution and phased removal of methadone is an acceptable method of detoxification for patients and therapists. The DRUG TEST Panel yields a positive result when the concentration of methadone in urine exceeds the detection level of 300 ng/mL.

Methamphetamine (MET) Methamphetamine is an addictive stimulant drug that strongly activates certain systems in the brain. Methamphetamine is closely related chemically to amphetamine, but the central nervous system effects of methamphetamine are greater. Methamphetamine is made in illegal laboratories and has a high potential for abuse and dependence. The drug can be taken orally, injected, or inhaled. Acute higher doses lead to enhanced stimulation of the central nervous system and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to methamphetamine include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, psychotic behavior, and eventually, depression and exhaustion. The effects of methamphetamine generally last 2-4 hours and the drug has a half-life of 9-24 hours in the body. Methamphetamine is excreted in the urine primarily as amphetamine, and oxidized and deaminated derivatives. However, 10-20% of methamphetamine is excreted unchanged. Thus, the presence of the parent compound in the urine indicates methamphetamine use. Methamphetamine is generally detectable in the urine for 1-4 days, depending on urine pH level. The DRUG TEST Panel yields a positive result when the methamphetamine in urine exceeds the detection level of 500 ng/mL.

Morphine (MOP) Opiate refers to any drug that is derived from the opium poppy, including the natural products, morphine and codeine, and the semi-synthetic drugs such as heroin. Opioid is more general, referring to any drug that acts on the opioid receptor. Opioid analgesics comprise a large group of substances which control pain by depressing the CNS. Large doses of morphine can produce higher tolerance levels, physiological dependency in users, and may lead to substance abuse. Morphine is excreted unmetabolized, and is also the major metabolic product of codeine and heroin. Morphine is detectable in the urine for 10-55 hours after an opiate dose. The DRUG TEST Panel yields a positive result when the concentration of morphine in urine exceeds the detection level of 300 ng/mL.

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