Test your health

PRIMA LAB SA
Viale Serfontana, 10
CH-6834 Morbio Inferiore - SWITZERLAND
support@primalabsa.ch
primalabsa.ch

EC REP QbD Rep5 BV
Groenenborgerlaan 16
2610 Wilrijk - Belgium

800017_IFU_11_9.0 03/2026 REF 800017-x

Registered trademarks

PRIMA
HOME TEST

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

HELICOBACTER PYLORI TEST

Self-test for the determination of anti Helicobacter pylori antibodies in whole blood samples

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Hualan Tianda Medical Instruments Co., Ltd., No.106 East Sonjiang Road, Huaiyin, Economic&Technological Development Zone, 223300 - Hualan City, Jiangsu - P. R. China

EC REP Romavix Sociedad Limitada
Calle de Almansa 55, 1D
Madrid 28039, Spain

EC REP Phoenix Innovative Healthcare Manufacturers Pvt. Ltd., 2nd Flr, Tower Street, Swatar, TTC Industrial Area, Mahape, Navi Mumbai 400 710 MH I India

SYMBOLS / SYMBOLE / SYMBOLES / SIMBOLOGIA

IVD In vitro diagnostic device In vitro Diagnostikum Dispositif médical de diagnostic in vitro Dispositivo medico-diagnostico in vitro	Temperature limits Temperaturbegrenzung Limites de température Limiti di temperatura	REF List number Katalognummer Code produit Codice prodotto
Read the instructions before use Lesen Sie die Gebrauchsanweisung Consultez le mode d'emploi Leggere attentamente le istruzioni per l'uso	Do not reuse Ne pas réutiliser Non riutilizzare	LOT Lot number Chargenbezeichnung Code du lot Lotto Prodotto
STERILE R Sterilized using irradiation Sterilisation durch Bestrahlung Méthode de stérilisation utilisant l'irradiation Sterilizzato mediante radiazioni	Sufficiency for <n> tests Ausreichend für <n> Tests Suffisant pour <n> tests Sufficiente per <n> test	CE marking CE-Kennzeichnung Marchio CE
Authorized Representative in the European Community Bevollmächtigter in der Europäischen Gemeinschaft Mandatario nella Comunità Europea	Expiry date (last day of the month) Verwendbar bis (letzter Tag des Monats) Utiliser jusqu'au (dernier jour du mois) Utilizzare entro (ultimo giorno del mese)	Legal manufacturer Hersteller Fabricant Fabbricante
Importer / Importeur / Importateur / Importatore		

ENGLISH HELICOBACTER PYLORI INFECTION AND SPECIFIC IgG

Helicobacter pylori is a bacterium whose ideal habitat is the human gastric mucosa. The infection is often asymptomatic, but it can sometimes cause gastritis and ulcers in the stomach or in the duodenum segment of the small intestine. The most common symptoms of these conditions are heartburn or pain in the upper part of the abdomen, especially if fasting or in the early morning, when the stomach is empty. Other associated symptoms include nausea, feeling full, reflux, loss of appetite, burping, weight loss and diarrhoea. Ulcers can sometimes bleed, causing anemia over long periods of time. In the long term, H. pylori infection is associated with an increase in the risk of gastric cancers. The simplest way to detect infection is by checking for the presence of specific antibodies (IgG) that are produced by the immune system after coming into contact with the bacterium and trying to eliminate it. The Helicobacter pylori Test can detect the presence of these antibodies in a few minutes.

PRINCIPLE OF THE TEST

Helicobacter pylori Test is an immunochromatographic test that detects the presence of anti-Helicobacter pylori IgG antibodies in the blood, using special monoclonal antibodies conjugated with gold and integrated into the reactive strip.

REFERENCES

1. Mati Moyat, Dominique Velin. Immune responses to Helicobacter pylori infection. World J. Gastroenterol. 2014 may 21; 20(19): 5583-5593IgE." Journal of Allergy and Clinical Immunology. 2014;133(2):589-91.
2. Khean-Lee Goh, Wah-Kheong Chan, Seiji Shiota and Yoshio Yamaoka. Epidemiology of Helicobacter pylori Infection and Public Health Implications. Helicobacter, 2011 Sep;16(0 1):1-9.
3. Kyichi Adachi, Tomoko Mishiro, Shino Tanaka and Yoshikazu Kinoshita. Analysis of negative result in serum anti-pylori IgG antibody test in cases with gastric mucosal atrophy. J. Clin Biochem Nutr.2016 Sep;59(2):145-148.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

* hermetically sealed aluminium pouch containing: 1 HELICOBACTER PYLORI TEST cassette 1 desiccant bag	* transparent plastic bag containing a pipette for collecting blood
* sterile lancets for self-drawing blood	* alcohol cleansing gauze
* vial with dropper tip containing sufficient HELICOBACTER PYLORI TEST diluent for 1 test	* 1 instructions for use leaflet

- Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
- Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
- The desiccant packet must not be used. Dispose of it with household waste without opening it.

PRECAUTIONS

1. Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
2. Keep the Test out of the reach of children.
3. Do not use the Test after the expiry date or if the package has been damaged.
4. Follow the procedure exactly, using only the specified quantities of blood and diluent.
5. Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
6. Use the test and lancing device once only.
7. The test is for external use only. DO NOT SWALLOW.
8. In vitro diagnostic device for individual use.
9. Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
10. After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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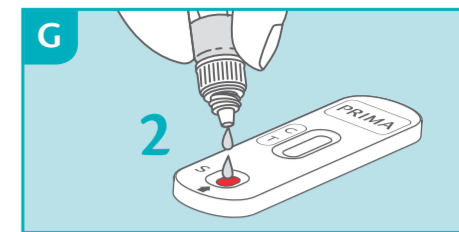
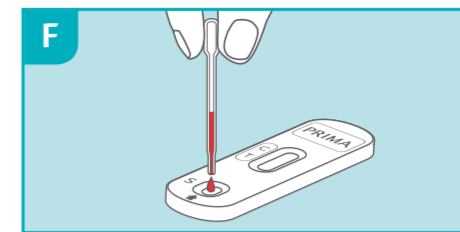
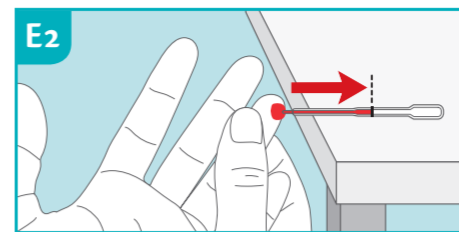
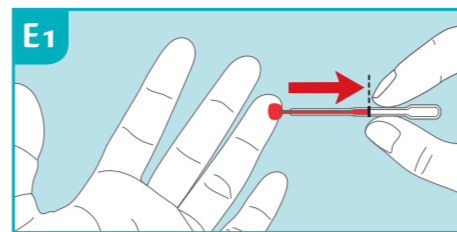
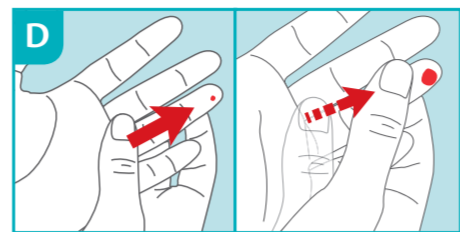
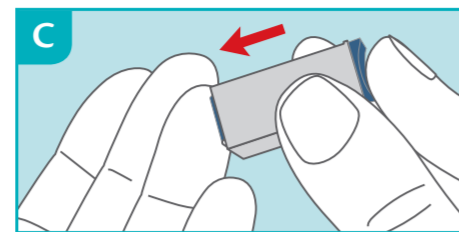
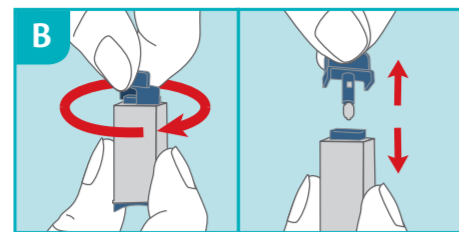
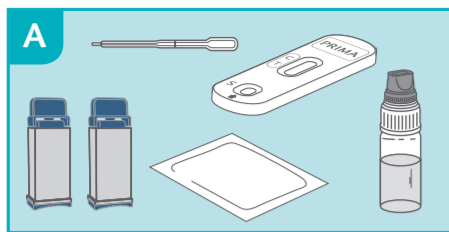
ENGLISH TEST PROCEDURE

- Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
Note: The use of warm water facilitates capillary blood collection as it induces vasodilation.
- Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag. Open the plastic packet containing the pipette. –**FIG. A**
- Carefully rotate the protective cap of the sterile lancet 360° without pulling it. Extract and discard the released cap. –**FIG. B**
- Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the phalanx, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip –**FIG. C** The tip of the lancet automatically retracts after use. If the lancet does not work properly, discard it and use the second one supplied. If the second one is not required, it can be disposed of without special precautions.
- Holding the hand down, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the phalanx to improve blood flow. –**FIG. D**
- Take the pipette **without pressing the bulb**. Two sampling methods are suggested:
–**FIG. E1:** hold the pipette horizontally **without pressing the bulb** place it in contact with the drop of blood, it will enter the pipette by capillarity. Move the pipette away when the black line is reached. If there is not enough blood, continue to massage the finger until the black line is reached.
–**FIG. E2:** place the pipette on a clean, flat surface with the tip protruding from the shelf, then place the drop of blood in contact with the pipette, it will enter by capillarity. If blood is not sufficient, continue massaging the finger until the blood has reached the black line.
Avoid, as far as possible, to continuously move the tip of the pipette away from the finger in order to prevent the formation of air bubbles.
- Place the blood collected with the pipette into the well indicated on the cassette (S) by pressing the pipette bulb. –**FIG. F**
- Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on). Deposit **2 drops** into the well indicated on the cassette (S), wait for 5 seconds between the first and second drop. –**FIG. G**
- Wait **5 minutes** and read the results referring to the next section for result interpretation.

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. **This result means there are anti-Hp antibodies in the blood and you must consult a physician.**

NEGATIVE RESULTS

A coloured band appears only under the C (control) sign. **This result means that the presence of anti-Hp antibodies cannot be detected in the blood.**

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new blood sample.

F.A.Q. – QUESTIONS AND ANSWERS

HOW DOES THE HELICOBACTER PYLORI TEST WORK? IgG antibodies are produced in high quantities by our body in response to infections. The Helicobacter pylori Test identifies class IgG antibodies specific for Helicobacter pylori and reveals if the concentration in the blood of these antibodies is 15 AU/mL. In some cases, samples with IgG concentrations between 10 AU/mL and 14.99 AU/mL can yield positive results.

WHEN CAN THE TEST BE USED? HELICOBACTER PYLORI TEST is recommended when the typical symptoms of Helicobacter pylori gastric infection are present, such as stomach pain, heartburn, nausea, feeling full, burping, loss of appetite, diarrhoea and weight loss. The Test can be carried out at any time of the day. If you are taking medication, please talk to your physician to check that the drugs you are taking do not alter the levels of circulating IgG.

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally split, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3. The pipette supplied allows users to be sure they have collected the right amount of blood.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the

result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the test band.

IS THE RESULT RELIABLE IF READ AFTER 5 MINUTES? No. The test must be read 5 minutes after the procedure is completed.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? A positive result means that the level of anti-Helicobacter pylori specific antibodies belonging to the IgG class is equal to or higher than 15 AU/mL. The test does not discriminate between ongoing infection or past infections, so consult a physician and show him/her the result of the test. Your physician will decide which other tests you should perform and advise you accordingly.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the level of anti-Helicobacter pylori specific antibodies belonging to the IgG class is lower than standard. If symptoms persist, consult your physician.

HOW ACCURATE IS THE HELICOBACTER PYLORI TEST? The Test is very accurate. Assessment reports show a concordance rate above 87% (CI 95%: 80.4 – 93.6%) with reference methods.

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