BIBLIOGRAPHY / LITERATURVERZEICHNIS / RÉFÉRENCES / BIBLIOGRAFIA / BIBLIOGRAFIA / BIBLIOGRAFIA / REFERENCER / REFERENCE

SYMBOLS / SIMBOLES / SIMBOLÓGIA / SIMBOLOS / SYMBOLE / SIMBOLO / SYMBOLER / SYMBOLE

CE marking / CE-Kennzeichnung / Marquage CE / Marchio CE / Marcado CE / Знак CE / CE-markering / CE-märkning / Značka CE

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PRIMA HOME TEST STREP A TEST

Self-test for the detection of group A beta-hemolytic Streptococcus in throat swab

EN - INSTRUCTIONS FOR USE STREPTOCOCCUS A

Group A beta-hemolytic Streptococcus (also known as Streptococcus pyogenes) is a Gram-positive bacterium that causes a variety of pathological conditions and complications...

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the symptoms were not caused by the Streptococcus A or that the sample did not contain a concentration of bacteria that was high enough to be detected by the test.

IS THE RESULT RELIABLE IF READ AFTER 10 MINUTES? NO. The test must be read 5 minutes after the procedure is completed and no later than 10 minutes from this time.

CAUTION! Read these instructions for use carefully before performing the test. The test is reliable if the instructions are followed correctly (reaction times when the sample is collected and the extraction instructions shown).

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PRECAUTIONS 1. Read these instructions for use carefully before performing the test. The test is reliable if the instructions are followed correctly (reaction times when the sample is collected and the extraction instructions shown).

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