
Test your health

PRIMA LAB SA
Viale Serfontana, 10
CH-6834 Morbio Inferiore - SWITZERLAND
support@primalabs.ch
primalabs.ch

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800662-1\_UFJ\_9\_0 03/2026
800662-1\_HSP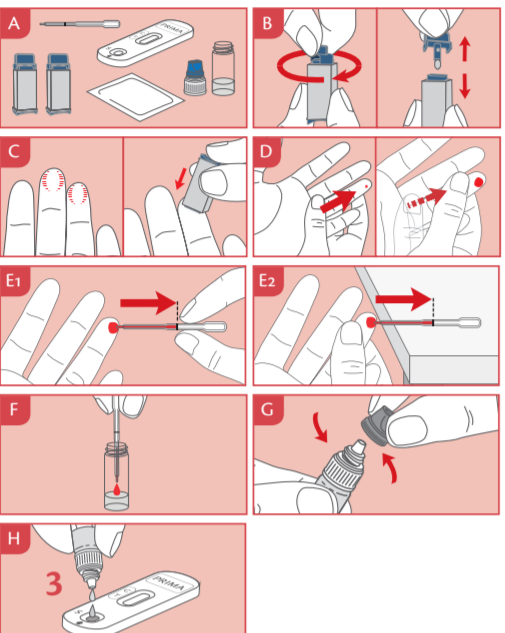

EN - INSTRUCTIONS FOR USE
WHAT IS HIV?
The human immunodeficiency virus (HIV) is a pathogen which attacks and suppresses the immune system by affecting in particular white blood cells.
The possible transmission pathways for HIV are:
• Sexual through sexual intercourse not protected by an efficient method of prevention (e.g. condoms).
• Blood: through transfusions of infected blood.
• Vertical: from mother to baby during pregnancy, at birth and, more rarely, through breastfeeding.
Without taking specific drugs, the HIV virus weakens the immune system leading to the acquired immunodeficiency syndrome (AIDS).
WHAT IS THE AIDS?
AIDS identifies an advanced clinical stage of HIV infection. It is a syndrome that can occur in people with HIV even several years after the infection, when the immune system's efficiency dramatically falls, and the body loses its ability to fight even the most common infections. The AIDS can be prevented by taking antiretroviral therapy at an early stage.
TEST PRINCIPLE & INTENDED USE
HIV 1/2 SELF-TEST is a self-test that is able to specifically detect the presence in a blood sample of anti-HIV-1 and/or 2 antibodies. If the sampled drop of blood contains anti-HIV antibodies, they are captured by the test producing a red-coloured line in the Test line region (T). Otherwise, if the sample does not contain such antibodies, no colour will appear in the Test line region (T). The absence of colour in the Test line region (T) is also possible when the levels of circulating antibodies are below the limit of detection, leading to false negative results, especially for HIV-1 Group O (prevalence of HIV 1 Group O worldwide: 1%). A red-coloured line will always appear in the Control line region (C) confirming that the test has correctly operated.
HOW DOES THE HIV 1/2 SELF-TEST WORK?
The test detects the presence of antibodies to HIV-1 and/or HIV-2 antigens and colloidal gold nanoparticles embedded in the test strip.
WHEN CAN THE TEST BE USED?
HIV 1/2 SELF-TEST can be performed at any time of the day. The test can be carried out by anyone who has been or suspects to have been exposed to the virus, people with HIV symptoms or anyone who wants to be aware of a possible infection. Antibody tests can take 23 to 90 days (window period) to detect HIV infection after an exposure. HIV 1/2 SELF-TEST is not for use in emergency situations.
WHAT IS THE DIFFERENCE BETWEEN HIV AND AIDS?
HIV is a virus that attacks the human immune system. Under normal circumstances, immune system cells defend the body against various pathogens - including viruses. However, HIV destroys exactly those cells that should protect the body against it. AIDS is a late consequence of untreated HIV infection. Being HIV positive and having AIDS is not the same thing. The HIV multiplies and damages the immune system until AIDS develops.
CAN THE RESULT BE NOT CORRECT?
The result is correct as long as the instructions are carefully followed. However, the result may not be correct if the device comes into contact with other liquids before use, if the amount of blood and/or diluent is insufficient, if the number of drops dispensed into the well is incorrect or if the reading time of the result is not respected. The plastic pipette supplied allows to be sure that the volume of collected blood is correct. False positive results (the test incorrectly indicates a positive result even if the anti-HIV antibodies are not present) may be due to Mycoplasma pneumoniae, Borrelia, Hepatitis C Virus (HCV), Human T-lymphotropic virus (HTLV), Epstein-Barr Virus (EBV), Human Herpes Virus 8 (HHV-8). The main consequence of a false positive result is that the person could experience a strong negative stress and would have to undergo a second unnecessary laboratory test to have the diagnosis confirmed. Due to the lack of samples with very high concentrations of antibodies, the manufacturer cannot exclude the possibility that very high concentrations of antibodies against HIV-1/2 in the blood sample could lead to false negative results (the test incorrectly indicates a negative result even if the anti-HIV antibodies are actually present).
HOW TO INTERPRET THE TEST IF THE COLOR AND INTENSITY OF THE TEST AND CONTROL LINES ARE DIFFERENT?
The colour and intensity of the lines are not relevant for the interpretation of the results. Refer to the indications and to the images in the section "Results Interpretation" to correctly evaluate the result.
IF READ AFTER 10 MINUTES, IS THE RESULT RELIABLE?
No. The test should read exactly 10 minutes after the end of the procedure. Results read after 10 minutes may not be correct (false positive results may appear, see question "Can the result be not correct" where the consequences of false positive results are explained).
WHAT SHOULD BE DONE IF THE RESULT IS POSITIVE?
If the result is positive, it means that antibodies to HIV 1 and/or HIV 2 are present in the blood sample. You should consult your physician and report the result.
WHAT SHOULD BE DONE IF THE RESULT IS NEGATIVE?
If the result is negative, the test did not identify any antibodies to HIV-1 and/or HIV-2. This result may mean that HIV infection has not occurred or is at an early stage where specific antibodies have not been yet developed. False negative results may be obtained with patients receiving antiretroviral or cognitive therapy (e.g. conditions of immunosuppression). If your result is negative but you think you have been exposed or you have symptoms, please contact your doctor to ensure that you are not in the window period (3 months), which is the time required for the development of a detectable amount of HIV antibodies. Please avoid any activity that may cause HIV transmission to others.

SYMBOLS / SYMBOLE / SYMBOLES / SIMBOLGIA / SYMBOLE / SIMBOLEN / SYMBOLE
IVD In vitro diagnostic device / In-Vitro-Diagnostikum / Dispositivo medico de diagnostico in vitro / Dispositivo medico-diagnostico in vitro / Wyrob diagnostyczny do diagnostyki in vitro / In-vitro-diagnostisch hulpmiddel / Enhet för in-vitro-diagnostik
STERILE Sterilisation using irradiation / Sterilization mediante radiación / Wyszczelnienie przez promieniowanie / Gesteriliseerd door middel van straling / Steriliseerd med stråling
Exp. date (last day of the month) / Verwendbar bis (letzter Tag des Monats) / Utiliser jusqu'au (dernier jour du mois) / Utilizzare entro (ultimo giorno del mese) / Termin waznosci (ostatni dzien miasnicy) / Vervaldatum (laatste dag van de maand) / Sista dag tillgänglig (sista dagen i månaden)
Legal manufacturer / Hersteller / Fabricant / Fabricatore / Produsător / Juridische fabrikant / Juridisk tilvareker
CE-marking / CE-Kennzeichnung / Marquage CE / Marchio CE / Znak CE / CE-märkning / CE-märking
Temperature limits / Temperaturbegrenzungen / Limites de températures / Limiti de temperatura / Ograniczenia temperatury / Temperaturavgränsningar / Temperaturgränser
Do not reuse / Nicht wiederverwenden / Ne pas réutiliser / Non riutilizzare / Ne utnyazovat ponovne / Niet opnieuw / Återanvänd inte
Sufficient for <-> Tests / Ausreichend für <-> Tests / Sufficient pour <-> tests / Sufficiente per <-> tests / Wystarczaco na następujacy liczbę testów: <-> / Noleidende voor <-> tests / Tillräckligt för <-> tester
List number / Katalognummer / Code produit / Codice prodotto / Nummer wyszczelnienia / Listnummer / Listnummer
Lot number / Chargenbezeichnung / Code du lot / Lotto prodotto / Nummer partii / Lotnummer / Partinummer
Authorized Representative in the European Community / Bevollmächtigter in der Europäischen Gemeinschaft / Mandatario nella Comunità Europea / Autorizovaný predstaviteľ na teritórium Všeobecných Európskych / Gemachtigdheden vertegenwoordiger in de Europese Gemeenschap / Autorizovaný reprezentant i Europejskiej wspólnocie
Importer / Importeur / Importatör / Importatore / Importer / Importör / Importör
MEDICAL DEVICES INSIDE THE KIT / MEDICINSKES GERÄTE INNERHALB DES KITS / DISPOSITIFS MÉDICAUX À L'INTÉRIEUR DU KIT / DISPOSITIVI MEDICI INTERNI AL KIT / WYROBY MEDYCZNE W ZESTAWIE / MEDISCHE HULPMIDDELEN IN DE KIT / MEDICINSKA ENHETER I KITTET
Huanan Tiande Medical Instruments Co., Ltd.
106 East Sunning Road, Huanan, Economic&Technological Development Zone, 223300 - P.R. China
Riomava Societas Limitada
Calle de Almansa 55, 11, Madrid 28039, Spain
Phenix Innovative Healthcare Manufacturers Pvt. Ltd.
E-209, 5th Mahaga Road, Electronic Zone, IITDC, TTD Industrial Area, Mahape, Navi Mumbai 401 710 MH India
Advena Ltd., Tower Business Centre, 2BR F1R, Tower Street, Svatkar, BKR 4013, Malta

HIV 1/2 SELF-TEST

Rapid self-test for the qualitative detection of antibodies to human immunodeficiency virus 1 and 2 (HIV1/HIV2) in whole blood
Eigen-Schnelltest für den qualitativen Nachweis von Antikörpern gegen das Humane Immundefizienzvirus 1 und 2 (HIV1/HIV2) in Vollblut
Autotest rapide de détection qualitative d'anticorps ciblant le virus de l'immunodéficience humaine 1 et 2 (VH1-/VH2-) dans le sang total
Self-Test Rapido per la rilevazione qualitativa degli anticorpi contro il virus dell'immunodeficienza umana di tipo 1 e 2 (HIV1/HIV2) in campioni di sangue intero
Szybki test do samodzielnego wykonania do jakościowego wykrywania przeciwciał przeciw ludzkiemu wirusowi niedoboru odporności 1 i 2 (HIV1/HIV2) w krwi pełnej
Snelle zelftest voor de kwalitatieve opsporing van antilichamen tegen het humane immunodeficientievirus 1 en 2 (HIV1/HIV2) in volledige
Schnabt självtest för kvalitativ detektion av antikroppar mot humant immunbristvirus 1 och 2 (HIV1/HIV2) i helblod

IF THE RESULT IS NEGATIVE, CAN I STOP HAVING PROTECTED SEX?
No. The HIV 1/2 SELF-TEST does not exclude other sexually transmitted disease (such as Herpes, Syphilis, Chlamydia, Gonorrhoea, Viral Hepatitis). Moreover, a negative result does not mean that your partner is not HIV infected. Condoms are the safest way to protect yourself and others.
WHAT IS THE ACCURACY OF HIV 1/2 SELF-TEST?
The test is very accurate. Evaluation reports show a sensitivity of 99.3% (95% CI: 98.2-99.7%), a specificity of 99.7 (95% CI: 99.2-99.9%) and an overall accuracy of 99.6% (95% CI: 99.2-99.8%) with respect to the laboratory standard test.
PRECAUTIONS
1. The HIV 1/2 SELF-TEST is not intended for therapeutic follow-up with patients receiving antiretroviral therapy.
2. Read these instructions for use carefully before testing. The Test is reliable if the instructions are carefully followed.
3. Do not use the test after the expiration date or if the packaging is damaged.
4. Do not use the Test after the expiration date or if the packaging is damaged.
5. Follow the procedure exactly, using only specified quantities of blood and diluent.
6. Store the Test components at a temperature of +4°C to +30°C. Do not freeze.
7. Use the Test and the sterile lancet only once.
8. The Test is for external use only. DO NOT SWALLOW.
9. Do not use the Test on children or people with haemophilia problems.
10. After use, dispose of all components according to local regulations, ask your pharmacist for advice.
11. This test is only able to detect HIV infection and cannot be used for the detection of other sexually transmitted infections.
12. People who use this self-test should consult their doctor prior to making any medical decisions.
13. If testing other people, treat blood samples as potentially infected.
14. In vitro diagnostic device for individual use.
CONTENT
1. Hermetically sealed aluminium pouch containing 1 HIV 1/2 SELF-TEST cassette, 1 desiccant bag. Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked. The desiccant packet must not be used. Dispose of it with household waste without opening it.
2. 1 vial with dropper tip contain of the HIV 1/2 SELF-DILUENT required for 1 test; 2 sterile lancets for self-drawing blood; 1 transparent plastic bag containing a pipette for collecting blood; 1 alcoholic cleansing gauze; 1 instructions for use leaflet.
3. All the mentioned components are packed in a protective plastic bag. Material required but not supplied: cotton wool, which can be used to remove residual blood drops after the puncture, a device to measure time (ie. timer, watch).
WHAT IS THE TEST?
The HIV 1/2 SELF-TEST only detects the presence of antibodies to human immunodeficiency virus 1 and 2 (HIV-1/HIV-2) in the blood sample, which is indicative of virus infection and subsequent immune response. HIV 1/2 SELF-TEST is not used to detect other sexually transmitted infections.
WHAT IS THE DIFFERENCE BETWEEN HIV AND AIDS?
HIV is a virus that attacks the human immune system. Under normal circumstances, immune system cells defend the body against various pathogens - including viruses. However, HIV destroys exactly those cells that should protect the body against it. AIDS is a late consequence of untreated HIV infection. Being HIV positive and having AIDS is not the same thing. The HIV multiplies and damages the immune system until AIDS develops.
CAN THE RESULT BE NOT CORRECT?
The result is correct as long as the instructions are carefully followed. However, the result may not be correct if the device comes into contact with other liquids before use, if the amount of blood and/or diluent is insufficient, if the number of drops dispensed into the well is incorrect or if the reading time of the result is not respected. The plastic pipette supplied allows to be sure that the volume of collected blood is correct. False positive results (the test incorrectly indicates a positive result even if the anti-HIV antibodies are not present) may be due to Mycoplasma pneumoniae, Borrelia, Hepatitis C Virus (HCV), Human T-lymphotropic virus (HTLV), Epstein-Barr Virus (EBV), Human Herpes Virus 8 (HHV-8). The main consequence of a false positive result is that the person could experience a strong negative stress and would have to undergo a second unnecessary laboratory test to have the diagnosis confirmed. Due to the lack of samples with very high concentrations of antibodies, the manufacturer cannot exclude the possibility that very high concentrations of antibodies against HIV-1/2 in the blood sample could lead to false negative results (the test incorrectly indicates a negative result even if the anti-HIV antibodies are actually present).
HOW TO INTERPRET THE TEST IF THE COLOR AND INTENSITY OF THE TEST AND CONTROL LINES ARE DIFFERENT?
The colour and intensity of the lines are not relevant for the interpretation of the results. Refer to the indications and to the images in the section "Results Interpretation" to correctly evaluate the result.
IF READ AFTER 10 MINUTES, IS THE RESULT RELIABLE?
No. The test should read exactly 10 minutes after the end of the procedure. Results read after 10 minutes may not be correct (false positive results may appear, see question "Can the result be not correct" where the consequences of false positive results are explained).
WHAT SHOULD BE DONE IF THE RESULT IS POSITIVE?
If the result is positive, it means that antibodies to HIV 1 and/or HIV 2 are present in the blood sample. You should consult your physician and report the result.
WHAT SHOULD BE DONE IF THE RESULT IS NEGATIVE?
If the result is negative, the test did not identify any antibodies to HIV-1 and/or HIV-2. This result may mean that HIV infection has not occurred or is at an early stage where specific antibodies have not been yet developed. False negative results may be obtained with patients receiving antiretroviral or cognitive therapy (e.g. conditions of immunosuppression). If your result is negative but you think you have been exposed or you have symptoms, please contact your doctor to ensure that you are not in the window period (3 months), which is the time required for the development of a detectable amount of HIV antibodies. Please avoid any activity that may cause HIV transmission to others.

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