Test your health

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

PRIMA®
HOME TEST

EARLY PREGNANCY TEST

Rapid Self-Test for the early detection of the hCG in human urine sample
Schneller Selbsttest zur frühen hCG-Bestimmung in Human-Urinproben
Auto-test rapide pour la détection précoce de hCG dans les urines humaines
Self-test rapido per la rilevazione precoce dell'hCG nelle urine umane

Registered trademarks.

820010_IFU_11_8.0 11/2024

REF 820010-x

ENGLISH

THE hCG HORMONE AND PREGNANCY

During the very first phases of pregnancy, the fertilised egg is carried from the Fallopian tubes to the uterus, where it implants after about seven days. In this phase, the tissue that will eventually form the placenta starts producing human chorionic gonadotropin, also known as the hCG hormone, which is essential for the development and continuation of the pregnancy. The early appearance and rapid increase in the concentration of the hCG hormone in the urine make it an excellent marker to confirm the beginning of a pregnancy. EARLY PREGNANCY TEST is an ultrasensitive rapid self-test which detects the presence of hCG in urine at a very low concentration (10 mIU/mL).

TEST-PRINCIPLE

EARLY PREGNANCY TEST is an immunochromatographic test that detects the hCG hormone through special gold-conjugated monoclonal antibodies included in the reactive strip.

PRECAUTIONS

1. Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
2. Keep the Test out of the reach of children.
3. Do not use the Test after the expiry date or if the package has been damaged.
4. Follow the indicated procedure and time strictly.
5. Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
6. Use the test once only.
7. The test is for external use only. DO NOT SWALLOW.
8. In vitro diagnostic device for individual use.
9. After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

F.A.Q. - QUESTIONS AND ANSWERS

HOW DOES EARLY PREGNANCY TEST WORK? The device specifically identifies the human hCG hormone and reveals if the concentration of that hormone in the urine is equal to or above 10 mIU/ml. In some cases, samples with hCG hormone concentration below this value (between 7.4 mIU/ml and 9.99 mIU/ml) can yield a positive result. The Test was calibrated on the WHO International Standard NIBSC 99/688.

WHEN CAN I TAKE THE TEST? The test can be performed three days before the day your period is due, it is suggested to use the first urine of the morning (discard the first flow).

CAN THE RESULTS BE INCORRECT? The correct results are obtained if the instructions are followed correctly. However, there may be incorrect results if moisture ends up on the device before the test is carried out. The results may very occasionally be false positives or false negatives if there are hormone metabolism disorders or if you are being treated for infertility with hCG-containing drugs. Other infertility treatments (such as clomiphene citrate), painkillers and hormone-based contraceptives (such as the pill) should not affect the result. However, if you have recently stopped taking hormone-based contraceptives or if you are having infertility treatment, your period may be irregular and cause you to take the test early. If you have recently been pregnant/miscarried, you can have false positive results because of the hCG hormone still in your system. In the case of unexpected or contradictory results, you should talk to your gynaecologist, because some health conditions may alter the test's results.

IF THE ABSORBING TIP IS LEFT IN THE URINE FOR MORE THAN 10 SECONDS, IS THE RESULT CORRECT? Yes, a few seconds more do not interfere with the result.

IF THE ABSORBING TIP IS LEFT IN THE URINE FOR LESS THAN 10 SECONDS, IS THE RESULT CORRECT? No, because the volume of the absorbed sample may not be enough.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive, the EARLY PREGNANCY TEST has identified the presence of the hCG hormone at a concentration equal to or higher than 10 mIU/ml. Speak to your gynaecologist. If the result is positive, a pregnancy may not be confirmed later as a result of an early miscarriage.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? If the result is negative, the hCG hormone levels in the urine are below 10 mIU/ml, which means that you are not pregnant or the concentration of hCG is too low to be detected by the test. If you tested early, test again when your period is expected. If this test gives you a negative result and you still have not had your period, consult your doctor.

CAN THE TEST BE USED AS BIRTH CONTROL? No, the test has not been designed for that purpose and it must not be used for that purpose.

HOW ACCURATE IS THE EARLY PREGNANCY TEST? Assessment reports show a 100% concordance rate (CI 95%:100%) with the reference methods.

REFERENCES

1. Stenman U.H., Alftan H.: "Determination of human chorionic gonadotropin." *Best Pract Res Clin Endocrinol Metab.* 2013 Dec;27(6):783-93.
2. Nwabuobi C., Arlier S., Schatz F., Guzeloglu-Kayisli O., Lockwood C.J. and Umit Ali Kayisli U.A.: "hCG: Biological Functions and Clinical Applications" *Int J Mol Sci.* 2017 Oct; 18(10): 2037.
3. Batzer FR. "Hormonal evaluation of early pregnancy", *Fertil. Steril.* 1980; 34(1): 1-13 2.

CONTENT

*The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * hermetically sealed aluminium pouches containing each: 1 device for the EARLY PREGNANCY TEST including a protective cap and 1 desiccant bag
- * 1 instructions for use leaflet

- Do not open the sealed aluminium pouch until just before performing the test. Take care to open it as marked.
- The desiccant bag must not be used. Dispose of it with household waste without opening it.
- Material required but not supplied: a device to measure time (i.e. timer, watch).

TEST PROCEDURE

- 1) Open the protective bag and take out the Test device. Dispose of the desiccant bag contained in the aluminium pouch.
- 2) Remove the protective cap - this exposes the absorbing tip. -**FIG. A** Rotate the device so that the results window faces up and the absorbing tip faces down.
- 3) Place the absorbing tip right under the flow of urine for at least 10 seconds. -**FIG. B1**
WARNING: Ensure no drops of urine splash onto the reading window. Alternatively (if easier), urine can be collected in a clean, dry, residue-free container (not supplied) and the absorbing tip (the entire oval) can be dipped into the urine for 10 seconds -**FIG. B2**
- 4) Place the protective cap back onto the absorbing tip. -**FIG. C**
- 5) Put the test stick onto a dry, flat surface with the results window facing up. -**FIG. D**

RESULTS INTERPRETATION


READ THE RESULT AFTER EXACTLY 5 MINUTES

The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the results window in the T (Test) and C (Control) regions. The T (Test) band may be less intense (lighter) than the C (Control) line. This means that the concentration of the hCG hormone in the urine is equal to or higher than 10 mIU/ml, meaning you are pregnant. Strong positives appear before the fifth minute.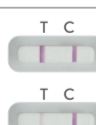

NEGATIVE RESULTS

A coloured line appears only in the C (Control) region. This means that the concentration of hCG hormone in the urine is below 10 mIU/ml and that you are not pregnant.

NON VALID RESULT

No line appears in the C (Control) region and it is impossible to interpret the result of the test, which must be considered not valid. You should repeat the test with a new EARLY PREGNANCY TEST and a new urine sample.

The most frequent causes of incorrect use are:

- a) the absorbing tip was immersed in the urine for too long or not long enough;
- b) drops of urine splashed onto the results window while the tip was under the urine flow.

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