Test your health

PRIMA LAB SA
Viale Serfontana, 10
CH-6834 Morbio Inferiore - SWITZERLAND
support@primalabsa.ch
primalabsa.ch

EC REP QbD RepS BV
Groenenborgerlaan 16
2610 Wilrijk - Belgium

820050_IFU_11_9.0 03/2026

REF 820050-x

Registered trademarks.

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

OVULATION LH TEST

Self-test for detecting the luteinizing hormone (LH) in urine

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PRIMA
HOME TEST

ENGLISH THE LUTEINIZING HORMONE AND OVULATION

During a woman's fertile age (from puberty to menopause), her reproductive system will go through a series of important structural and functional changes every month. This occurs as a result of the continuous variations in the levels of oestrogens and progesterone. These hormones are regulated by the hypothalamic-pituitary-gonadal axis and are responsible for the menstrual cycle, which has 4 phases, i.e. menstrual phase, follicular phase, ovulation and luteal phase. During the ovulation phase, which occurs around day 12/13 of the menstrual cycle, the level of oestrogens in the blood is so high that the pituitary gland starts to produce the luteinizing hormone (LH). This leads to follicle rupture and the release of the egg, which is ready to be fertilised. The egg can be fertilised for the 24 hours following release.

Measuring LH levels in the urine is therefore an effective way to optimise the chances of conception. The OVULATION LH TEST allows you to detect a level of LH above 25 mIU/ml and therefore identify the days you are most fertile.

TEST PRINCIPLE

OVULATION LH TEST is an immunochromatographic test that detects the luteinizing hormone (LH) through special gold-conjugated monoclonal antibodies included in the reactive strip.

REFERENCES


1. Su HW, Yi YC, Wei TY, Chang TC, Cheng CM "Detection of ovulation, a review of currently available methods" *Bioeng Transl Med.* 2017 May 16;2(3):238-246.
2. Leiva RA, Bouchard TP, Abdullah SH, Ecochard R.: "Urinary Luteinizing Hormone Tests: Which concentration threshold best predicts ovulation?" *Front public health* 2017, 5: 320.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * hermetically sealed aluminium pouches containing each:
1 device for the OVULATION LH TEST including a protective cap
1 desiccant bag

- * 1 instructions for use leaflet

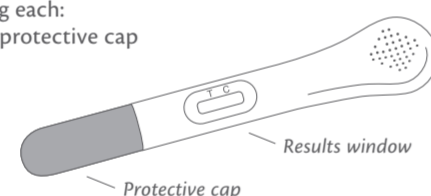- Do not open the sealed aluminium pouch until just before performing the test. Take care to open it as marked.
- The desiccant bag must not be used. Dispose of it with household waste without opening it.
- Material required but not supplied: a device to measure time (i.e. timer, watch).

PRECAUTIONS

1. Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
2. Keep the Test out of the reach of children.
3. Do not use the Test after the expiry date or if the package has been damaged.
4. Follow the indicated procedure and time strictly.
5. Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
6. Use the test once only.
7. The test is for external use only. DO NOT SWALLOW.
8. In vitro diagnostic device for individual use.
9. After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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




Begin of the test
 Testbeginn
 Debut du test
 Inizio test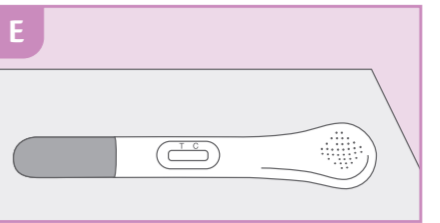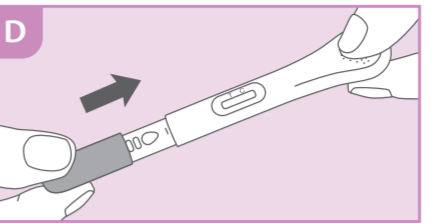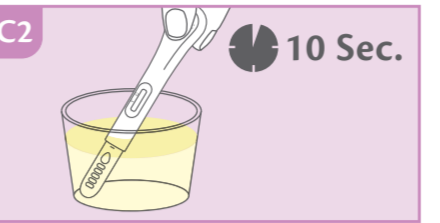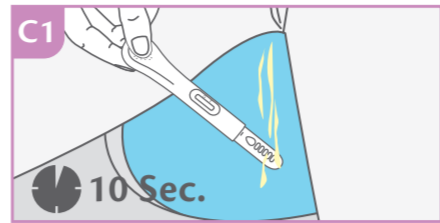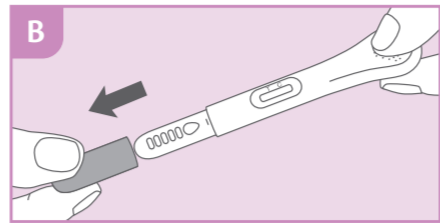

WHEN TO PERFORM THE TEST?

To understand when to perform the test you must know how long your cycle lasts. The length of a cycle is from the first day your period begins and the day the next period begins. Use the table 1 to establish the day in which you should start testing. The first day of your period is shown as day 1. If your period is irregular, consider the duration of the shortest cycle in the last 6 months. Refer to the table to establish the day in which you should start testing. If your cycle is shorter than 21 days or longer than 40, please talk to your physician.

TAB. 1

CYCLE DURATION	21-22 days	23-24 days	25 days	26 days	27 days	28 days	29 days	30 days	31 days	32 days	33 days	34 days	35 days	36 days	37 days	38 days	39 days	40 days
BEGIN OF THE TEST	6th day	7th day	8th day	9th day	10th day	11th day	12th day	13th day	14th day	15th day	16th day	17th day	18th day	19th day	20th day	21st day	22nd day	23rd day

EXAMPLE: the average duration of your cycle is 28 days. Your last period started on the 3rd of the month. The table 1 shows that you should count 11 days from the 3rd day of the month (included), so the first test should be done on the 14th day of that month. -FIG. A

TEST PROCEDURE

- 1) Open the protective bag and take out the Test device. Dispose of the desiccant packet contained in the aluminium bag.
- 2) Remove the protective cap - this exposes the absorbing tip. -FIG. B. Rotate the device so that the results window faces up and the absorbing tip faces down.
- 3) Place the absorbing tip right under the flow of urine for at least 10 seconds. -FIG. C1. WARNING: Ensure no drops of urine splash onto the reading window. Alternatively (if easier), urine can be collected in a clean, dry, residue-free container (not supplied) and the absorbing tip can be dipped into the urine for 10 seconds -FIG. C2
- 4) Place the protective cap back onto the absorbing tip. -FIG. D
- 5) Put the test stick onto a dry, flat surface with the results window facing up. -FIG. E

RESULTS INTERPRETATION

READ THE RESULT AFTER EXACTLY 5 MINUTES
 The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the results window in the T (Test) and C (Control) regions. The T (Test) band may be less intense (lighter) than the C (Control) line. This means that the concentration of the luteinizing hormone (LH) in the urine is equal to or higher than 25 mIU/ml. In this case you are reaching the ovulation peak and the chances of conceiving have increased.

5 min.

NEGATIVE RESULT

A coloured line appears only in the C (Control) region. This means that the concentration of luteinizing hormone (LH) in the urine is below 25 mIU/ml and that you have not reached the ovulation peak. Daily testing is recommended.

NON VALID RESULT

No line appears in the C (Control) region and it is impossible to interpret the result of the test, which must be considered not valid. You should repeat the test with a new OVULATION LH TEST and a new urine sample. The most frequent causes of incorrect use are:
 a) the absorbing tip was immersed in the urine for too long or not long enough;
 b) drops of urine splashed onto the results window while the tip was under the urine flow.

F.A.Q. - QUESTIONS AND ANSWERS

HOW DOES THE OVULATION LH TEST WORK? The device specifically identifies the human luteinizing hormone (LH) and reveals if the concentration of that hormone in the urine is equal to or above 25 mIU/ml. In some cases, samples with LH concentration below this value (between 22 mIU/ml and 24,99 mIU/ml) can yield a positive result. The Test was calibrated on the 3rd International Standard NIBSC 81/535.

AT WHAT TIME OF THE DAY SHOULD THE TEST BE CARRIED OUT? The test can be carried out at any time of the day, but some women have the best results with urine collected in the afternoon. If necessary, continue testing every day at the same time until you find the LH peak. Do not use with early morning urine.

CAN THE RESULTS BE INCORRECT? The correct results are obtained if the instructions are followed correctly. However, there may be incorrect results if moisture ends up on the device before the test is carried out. Additionally, in rare cases, the results may be erroneously positive or negative in women who are suffering from hormonal metabolic disorders. In the case of unexpected or contradictory results, you should talk to your gynaecologist, because some health conditions may alter the test's results.

IF THE ABSORBING TIP IS LEFT IN THE URINE FOR MORE THAN 10 SECONDS, IS THE RESULT CORRECT? Yes, a few seconds more do not interfere with the result.

IF THE ABSORBING TIP IS LEFT IN THE URINE FOR LESS THAN 10 SECONDS, IS THE RESULT CORRECT? No, because the volume of the absorbed sample may not be enough.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? Increase the frequency of sexual intercourse because there is a greater chance of conceiving.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? If the result is negative, the levels of LH in the urine are below 25 mIU/ml, which means the chances of conceiving are low. You should repeat the test every day to identify the increase in hormone concentration.

CAN THE TEST BE USED AS BIRTH CONTROL? No, the test has not been designed for that purpose and it must not be used for that purpose.

DOES GOING OFF ORAL CONTRACEPTIVES (THE PILL) INTERFERE WITH THE RESULTS? For some time after you stopped taking oral contraceptives, your periods may be irregular, which could interfere with the results.

HOW ACCURATE IS THE OVULATION LH TEST? Assessment reports show a concordance rate above 98% (CI 95%: 94 - 100%) with reference methods.

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