## Test your pet

**PRIMA LAB SA**  
Viale Serfontana, 10  
CH-6834 Morbio Inferiore - SWITZERLAND  
support@primalabsa.ch  
primalabsa.ch

PET100\_IFU\_11\_9.0 03/2026 [REF] PET100

Registered trademarks.

INSTRUCTIONS FOR USE  
GEBRAUCHSANWEISUNG  
NOTICE D'UTILISATION  
ISTRUZIONI PER L'USO

## GIARDIA PET TEST

Rapid test for the qualitative identification of *Giardia lamblia* antigens in animal faeces

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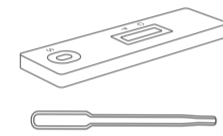
**PRIMA**  
HOME PET

### ENGLISH PRINCIPLE OF THE TEST

GIARDIA PET TEST is a lateral flow immunochromatographic rapid assay for the qualitative identification of *Giardia lamblia* antigens in animal faeces. During testing, a specimen is applied into the sample well on the cassette test. *Giardia* antigens, if present in the specimen, react with anti-*Giardia* antibodies coated colloidal gold particles in the test strip. In case of positive result two coloured bands will appear in the reading window by the T (Test) and C (Control) signs. If the specimen does not contain *Giardia* antigens a colored line will appear in the C (Control) area, showing a negative result. As a procedural control, a colored line will always appear in the C (Control) area, indicating that proper volume of specimen has been added and the test has been done correctly. It is not possible to read the result if no coloured line appears or there is only one under the T (Test) sign.

### CONTENT

\* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- \* hermetically sealed aluminium pouch containing:
  - 1 GIARDIA PET TEST cassette
  - 1 desiccant bag
  - 1 pipette for collecting the specimen

- \* vial with green screw cap, with collection stick and dropper, containing enough GIARDIA PET TEST diluent required for 1 test

- \* 1 instructions for use leaflet

- Material required but not supplied: specimen collection containers, a device to measure time (i.e. timer, watch).
- Do not open the sealed aluminium pouch until just before performing the test. Take care to open it as marked.
- The desiccant bag must not be used. Dispose of it with household waste without opening it.

### STORAGE AND STABILITY

Store the sealed aluminium pouch at room temperature or refrigerated (2-30 °C). The test is stable through the expiry date printed on the sealed pouch. The test cassette must remain in the sealed pouch until use. DO NOT FREEZE. Do not use beyond the expiry date.

### PRECAUTIONS

1. Do not use after expiry date.
2. Handle all specimens as if they contain infectious agents. Observe established precautions against microbiological hazards throughout testing and follow the standard procedures for proper disposal of specimens.
3. Wear disposable gloves when specimens are being tested.
4. Humidity and temperature can adversely affect results.
5. Do not remove test cassette from its pouch until immediately before use.
6. Do not reuse the test kit.
7. Do not mix components from different lot and products.

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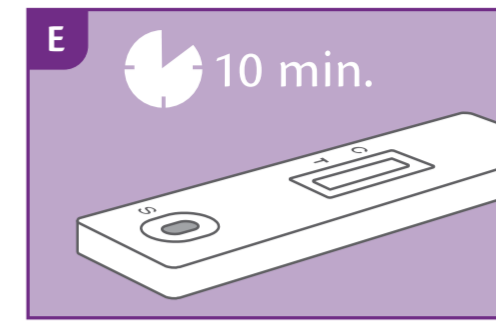
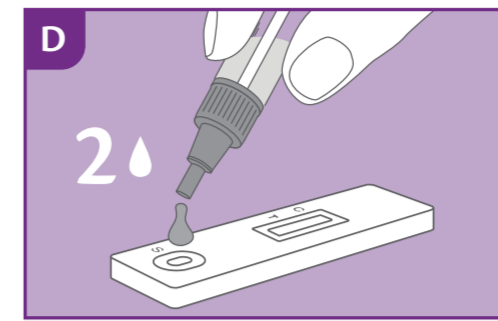
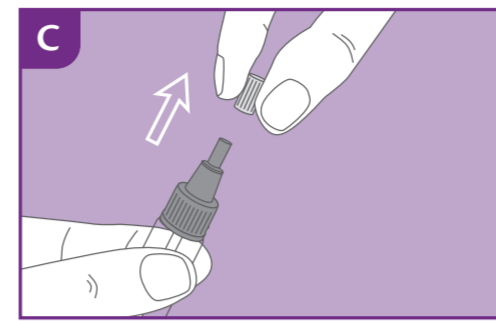
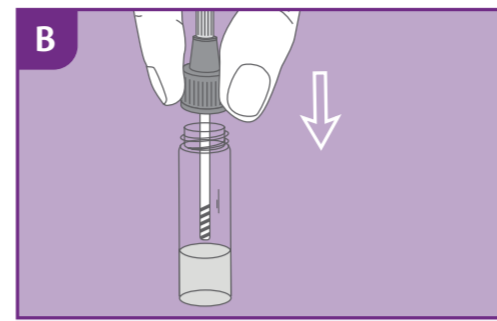
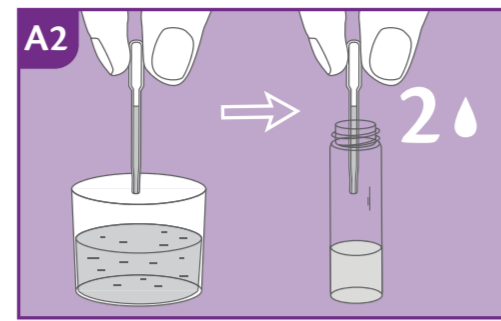
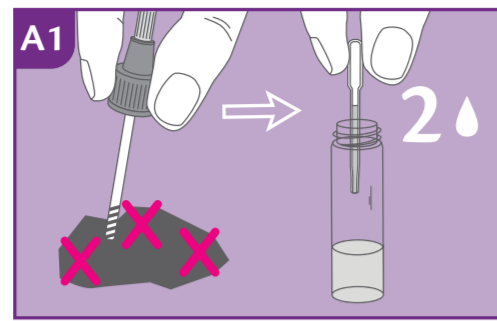
## ENGLISH TEST PROCEDURE

Allow the specimen and/or controls to reach room temperature (25 °C) before testing.

- 1) Collect a sufficient quantity of faeces (1-2ml or 1-2g) in a clean and dry container. Best results will be obtained if the assay is performed within 6 hours after sampling. Specimen collected may be stored for 3 days at 2-8°C if not tested within this time. For long term storage, specimen should be kept below -20°C of temperature.

### Collecting the sample:

**Solid Specimen:** Unscrew the vial's green cap and extract the faeces collecting-stick, then dip it into faeces in three different points, in order to collect approximately 50 mg of sample (equivalent to 1/4 of a pea). Do not scoop the faecal specimen. -FIG. A1

**Liquid Specimen:** Hold the dropper vertically, aspirate faecal specimen and then transfer 2 drops of the liquid one (approximately 80 µL) into the sample vial containing the faeces collecting-stick. -FIG. A2

Tighten the cap onto the vial, then shake it vigorously to mix the specimen. Leave the vial for reaction for 2 minutes. -FIG. B

- 2) Remove the test cassette from the aluminium pouch and use it as soon as possible.
- 3) Hold the specimen collection tube upright and unscrew the tip of the vial. -FIG. C Leave 2 drops of the extracted specimen (approximately 80 µL) to the specimen well of the test cassette, then start the timer. -FIG. D Avoid the formation of air bubbles.
- 4) Read the results at 10 minutes after dispensing the specimen. Do not read results after 20 minutes. -FIG. E

## RESULTS INTERPRETATION

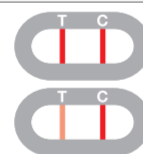
### READ THE RESULT EXACTLY AFTER 10 MINUTES

The colour intensity of the lines is not relevant for the interpretation of the Test's result.

### POSITIVE RESULT

Two coloured lines appear in the reading window by the T (Test) and C (Control) signs. The T line may be less intense (lighter) than the C one.


### NEGATIVE RESULTS

A coloured line appears only under the C (Control) sign.

### NON VALID RESULT

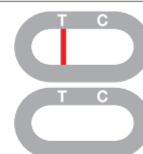If no lines appear or if there is only one under the T (Test) sign, it means that it was not possible to read the result of the test, which must be considered as not valid. It is recommended to repeat the test with a new GIARDIA PET TEST and new samples of faeces.

### LIMITATIONS

This is an acute-phase screening test. All results should be considered with other clinical information available from veterinarian.

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