



AMNIOCHECK

Amniotic leak self-test detector



GREEN/BLUE
Probable amniotic fluid leak



YELLOW
Urine leak



WHAT IS AMNIOCHECK

AMNIOCHECK is designed to detect any minute (100 microlitre) leak of amniotic fluid, differentiating it from urine by detecting the pH level.

WHO ARE THE INTENDED USERS

- Gynecologists' clinics or Delivery rooms: for amniotic fluid leakage screening or verification, especially for non-continuous leakage.
- Pregnant women at home: women with normal pregnancies experiencing unexplained wetness as well as by those at high risk of Premature Rupture of Membrane (PROM) and Preterm Premature Rupture of Membrane (PPROM).

Available for home use and professional use.


WHY - BENEFITS

Non invasive method, easy to use; early diagnosis of PROM and PPRM; reduction of unnecessary speculum examination; reduction of time spent in hospital; reduced risk of infection from speculum examination (particularly if repeat examinations are required); incidental detection of possible vaginal infection; reduction in staff time and hospital bed use.

HOW TO USE IT

Attach the panty liner to the underwear; the panty liner has a central polymer-embedded strip that turns blue-green on contact with fluid. Wear the panty liner as a normal panty liner till feel wetness (do not exceed 12 hours). At the first sense of vaginal wetness, remove the panty liner and wait up to 15 minutes and check for a colour change on the panty liner.

The detection panty liner is specific to vaginal fluid, and if contaminated with urine it will not give a positive result.

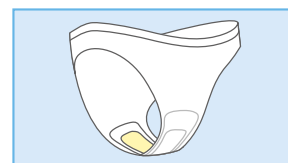

CONTENT: 5 aluminum pouches containing 1 panty liner each; 1 instructions for use leaflet.



VIDEO
TUTORIAL

REF	DESCRIPTION	NUMBER OF TESTS	SHELF LIFE
100088-5	AMNIOCHECK	5 TESTS	48 MONTHS

primalabsa.ch